

**PRODUCT SECURITY PATCH LIST
FOR
NUCLEAR MEDICINE EQUIPMENT
(2G984-0039EN)**

CANON MEDICAL SYSTEMS CORPORATION

Trademarks

Windows® is a registered trademark of Microsoft Corporation in the United States and other countries.

This document may also include trademarks or registered trademarks of other companies.

Note that the trademark symbol "™" and the registered trademark symbol "®" may or may not be used in this document.

IMPORTANT!

1. No part of this document may be copied or reprinted, in whole or in part, without written permission.
2. The contents of this document are subject to change without prior notice and without legal obligation.
3. The contents of this document are correct to the best of our knowledge. Please inform us of any ambiguous or erroneous descriptions, missing information, etc.

REVISION RECORD					
REV.	DATE (YYYY-MM)	REASON /AUTHOR	PAGE CHANGED	SER. No.	DOC. PRODUCT.
INI.	2025-03	Saito	-----		NASU

Security Patch Package name

SFDX60-07064_202502NM_hotfix_win10_LTSC2019_x64_enu_ProcWS

Applicable Products for the Security Patch

GCA-9300R	V4.1 SP0001J
-----------	--------------

Steps to Apply the Security Patch

Please follow the instruction of applying patches defined by each of the product Security Patch Package. For details, please contact the customer service for each system.

Verified Security Patch List

KB	MSRC	explanation(en-us)	URL/File Name	url
KB2538242_x86	n/a	Microsoft Visual C++ 2005 Service Pack 1 Redistributable Package MFC Security Update	vc redistrib_x86.exe	https://catalog.s.download.windowsupdate.com/msdownload/update/software/secu/2011/06/vcredist_x86_b8fab0bb7f62a24ddfe77b19cd9a1451abd7b847.exe
KB2538242_x64	n/a	Microsoft Visual C++ 2005 Service Pack 1 Redistributable Package MFC Security Update	vc redistrib_x64.exe	https://catalog.s.download.windowsupdate.com/msdownload/update/software/secu/2011/06/vcredist_x64_ee916012783024dac67fc606457377932c826f05.exe
KB2565063_x86	MS11-025	Microsoft Visual C++ 2010 Service Pack 1 Redistributable Package MFC Security Update	vc redistrib_x86.exe	https://catalog.s.download.windowsupdate.com/msdownload/update/software/secu/2011/07/vcredist_x86_28c54491be70c38c97849c3d8cfbffd0d3c515cb.exe
KB2565063_x64	MS11-025	Microsoft Visual C++ 2010 Service Pack 1 Redistributable Package MFC Security Update	vc redistrib_x64.exe	https://catalog.s.download.windowsupdate.com/msdownload/update/software/secu/2011/07/vcredist_x64_15d032d669078aa6f0f7fd1cbf4115a070bd034d.exe
VS2012_x86	n/a	Visual C++ Redistributable for Visual Studio 2012 Update 4	vc redistrib_x86.exe	https://download.microsoft.com/download/1/6/B/16B06F60-3B20-4FF2-B699-5E9B7962F9AE/VSU_4/vcredist_x86.exe
VS2012_x64	n/a	Visual C++ Redistributable for Visual Studio 2012 Update 4	vc redistrib_x64.exe	https://download.microsoft.com/download/1/6/B/16B06F60-3B20-4FF2-B699-5E9B7962F9AE/VSU_4/vcredist_x64.exe
KB3138367_x86	n/a	Visual C++ Redistributable Packages for Visual Studio 2013	vc redistrib_x86.exe	https://download.microsoft.com/download/2/E/6/2E61CFA4-993B-4DD4-91DA-3737CD5CD6E3/vcredist_x86.exe

KB	MSRC	explanation(en-us)	URL/File Name	url
KB3138367_x64	n/a	Visual C++ Redistributable Packages for Visual Studio 2013	vcredist_x64.exe	https://download.microsoft.com/download/2/E/6/2E61CFA4-993B-4DD4-91DA-3737CD5CD6E3/vcredist_x64.exe
VS2019_x86	n/a	Microsoft Visual C++ 2015-2019 Redistributable (x86)	vcredist_x86.exe	https://download.visualstudio.microsoft.com/download/pr/0c1cfec3-e028-4996-8bb7-0c751ba41e32/1abed1573f36075bdfdc538a2af00d37/vc_redist.x86.exe
VS2019_x64	n/a	Microsoft Visual C++ 2015-2019 Redistributable (x64)	vcredist_x64.exe	https://download.visualstudio.microsoft.com/download/pr/cc0046d4-e7b4-45a1-bd46-b1c079191224/9c4042a4c2e6d1f661f4c58cf4d129e9/vc_redist.x64.exe
KB5005112	n/a	2021-08 Servicing Stack Update for Windows 10 Version 1809 for x64-based Systems (KB5005112)	windows10.0-kb5005112-x64_81d09dc6978520e1a6d44b3b15567667f83eba2c.msu	https://catalog.s.download.windowsupdate.com/d/msdownload/update/software/secu/2021/08/windows10.0-kb5005112-x64_81d09dc6978520e1a6d44b3b15567667f83eba2c.msu
KB5052000	n/a	2025-02 Cumulative Update for Windows 10 Version 1809 for x64-based Systems (KB5052000)	windows10.0-kb5052000-x64_4d5c653ed24d769894ed1a2855d1c59fa70135af.msu	https://catalog.s.download.windowsupdate.com/c/msdownload/update/software/secu/2025/02/windows10.0-kb5052000-x64_4d5c653ed24d769894ed1a2855d1c59fa70135af.msu
KB5049615	n/a	2025-01 Cumulative Update for .NET Framework 3.5 and 4.8 for Windows 10 Version 1809 for x64 (KB5049615)	windows10.0-kb5049615-x64-ndp48_17bbac9f33539f6515297a7e2183d20b3b165424.msu	https://catalog.s.download.windowsupdate.com/d/msdownload/update/software/secu/2024/12/windows10.0-kb5049615-x64-ndp48_17bbac9f33539f6515297a7e2183d20b3b165424.msu
KB890830_v5.132	n/a	Windows Malicious Software Removal Tool x64 - v5.132 (KB890830)	windows-kb890830-x64-v5.132_99e2f6f80f1b15eaff192caff421e2c4c139796d.exe	https://catalog.s.download.windowsupdate.com/d/msdownload/update/software/uprl/2025/02/windows-kb890830-x64-v5.132_99e2f6f80f1b15eaff192caff421e2c4c139796d.exe



LEGAL MANUFACTURER

CANON MEDICAL SYSTEMS CORPORATION

1385, SHIMOISHIGAMI, OTAWARA-SHI, TOCHIGI 324-8550, JAPAN