

**PRODUCT SECURITY PATCH LIST
FOR
DIAGNOSTIC ULTRASOUND SYSTEM
(2G984-0068EN)**

CANON MEDICAL SYSTEMS CORPORATION

Trademarks

Windows® is a registered trademark of Microsoft Corporation in the United States and other countries.

This document may also include trademarks or registered trademarks of other companies.

Note that the trademark symbol "™" and the registered trademark symbol "®" may or may not be used in this document.

IMPORTANT!

1. No part of this document may be copied or reprinted, in whole or in part, without written permission.
2. The contents of this document are subject to change without prior notice and without legal obligation.
3. The contents of this document are correct to the best of our knowledge. Please inform us of any ambiguous or erroneous descriptions, missing information, etc.

REVISION RECORD					
REV.	DATE (YYYY-MM)	REASON /AUTHOR	PAGE CHANGED	SER. No.	DOC. PRODUCT.
INI.	2025-05	Anzai	-----		NASU

Security Patch Package name

SPZM37-20114_202505UL_hotfix_win10_LTSB2016_x64_enu

Applicable Products for the Security Patch

TUS-AI	V2.5 SP0000* - V2.5 SP0010*
	V3.0 SP0000* - V3.0 SP0019*
	V3.1 SP0000* - V3.1 SP0019*
	V4.0 SP0000* - V4.0 SP0019*
CUS-AA550/450	V1.3 SP0001* - V1.3 SP0013*
	V2.0 SP0000* - V2.0 SP0019*
CUS-AA000	V1.0 SP0001* - V1.0 SP0019*

Steps to Apply the Security Patch

Please follow the instruction of applying patches defined by each of the product Security Patch Package. For details, please contact the customer service for each system.

Verified Security Patch List

KB	MSRC	explanation(en-us)	URL/File Name	url
KB2565063_x86	MS11-025	Security Update for Microsoft Visual C++ 2010 Service Pack 1 Redistributable Package (KB2565063)	vc_redist_x86.exe	https://catalog.s.download.windowsupdate.com/msdownload/update/software/secure/2011/07/vcredist_x86_28c54491be70c38c97849c3d8cfbfdd0d3c515cb.exe
KB2565063_x64	MS11-025	Security Update for Microsoft Visual C++ 2010 Service Pack 1 Redistributable Package (KB2565063)	vc_redist_x64.exe	https://catalog.s.download.windowsupdate.com/msdownload/update/software/secure/2011/07/vcredist_x64_15d032d669078aa6f0f7fd1cbf4115a070bd034d.exe
VS2012_x86	n/a	Visual C++ Redistributable for Visual Studio 2012 Update 4	vc_redist_x86.exe	https://download.microsoft.com/download/1/6/B/16B06F60-3B20-4FF2-B699-5E9B7962F9AE/VSU_4/vcredist_x86.exe
VS2012_x64	n/a	Visual C++ Redistributable for Visual Studio 2012 Update 4	vc_redist_x64.exe	https://download.microsoft.com/download/1/6/B/16B06F60-3B20-4FF2-B699-5E9B7962F9AE/VSU_4/vcredist_x64.exe
VS2013_x86	n/a	Microsoft Visual C++ 2013 Update 5 Redistributable Package	vc_redist_x86.exe	https://download.microsoft.com/download/c/c/2/cc2df5f8-4454-44b4-802d-5ea68d086676/vcredist_x86.exe
VS2013_x64	n/a	Microsoft Visual C++ 2013 Update 5 Redistributable Package	vc_redist_x64.exe	https://download.microsoft.com/download/c/c/2/cc2df5f8-4454-44b4-802d-5ea68d086676/vcredist_x64.exe
VS2015_x86	n/a	Microsoft Visual C++ 2015 Redistributable Update 3 RC	vc_redist.x86.exe	https://download.microsoft.com/download/0/6/4/064F84EA-D1DB-4EAA-9A5C-CC2F0FF6A638/vc_redist.x86.exe

KB	MSRC	explanation(en-us)	URL/File Name	url
VS2015_x64	n/a	Microsoft Visual C++ 2015 Redistributable Update 3 RC	vc_redist.x64.exe	https://download.microsoft.com/download/0/6/4/064F84EA-D1DB-4EAA-9A5C-CC2F0FF6A638/vc_redist.x64.exe
KB5058524	n/a	2025-05 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems (KB5058524)	windows10.0-kb5058524-x64_d9c1dcfba3fc77d9e876d463f9df533c416ca2be.msu	https://catalog.s.download.windowsupdate.com/c/msdownload/update/software/secu/2025/05/windows10.0-kb5058524-x64_d9c1dcfba3fc77d9e876d463f9df533c416ca2be.msu
KB5058383	n/a	2025-05 Cumulative Update for Windows 10 Version 1607 for x64-based Systems (KB5058383)	windows10.0-kb5058383-x64_f2e34deb03d8284d2bc746589ec83fd20747edd0.msu	https://catalog.s.download.windowsupdate.com/c/msdownload/update/software/secu/2025/05/windows10.0-kb5058383-x64_f2e34deb03d8284d2bc746589ec83fd20747edd0.msu
KB890830_v5.133	n/a	Windows Malicious Software Removal Tool x64 - v5.133 (KB890830)	windows-kb890830-x64-v5.133.exe	https://catalog.s.download.windowsupdate.com/c/msdownload/update/software/uprll/2025/05/windows-kb890830-x64-v5.133_520f72b0ed0a3b1610e49a9ffc58aa497ed27c59.exe
KB5019182	n/a	2023-04 Update for Windows 10 Version 1607 for x64-based Systems (KB5019182)	windows10.0-kb5019182-x64_9dda0f01f337746913602d2c7269f5aa9cea9235.msu	https://catalog.s.download.windowsupdate.com/d/msdownload/update/software/updt/2023/04/windows10.0-kb5019182-x64_9dda0f01f337746913602d2c7269f5aa9cea9235.msu



LEGAL MANUFACTURER

CANON MEDICAL SYSTEMS CORPORATION

1385, SHIMOISHIGAMI, OTAWARA-SHI, TOCHIGI 324-8550, JAPAN