

**PRODUCT SECURITY PATCH LIST
FOR
DIGITAL RADIOGRAPHY SYSTEM
(2G984-0095EN)**

CANON MEDICAL SYSTEMS CORPORATION

Trademarks

Windows® is a registered trademark of Microsoft Corporation in the United States and other countries.

This document may also include trademarks or registered trademarks of other companies.

Note that the trademark symbol "™" and the registered trademark symbol "®" may or may not be used in this document.

IMPORTANT!

1. No part of this document may be copied or reprinted, in whole or in part, without written permission.
2. The contents of this document are subject to change without prior notice and without legal obligation.
3. The contents of this document are correct to the best of our knowledge. Please inform us of any ambiguous or erroneous descriptions, missing information, etc.

REVISION RECORD					
REV.	DATE (YYYY-MM)	REASON /AUTHOR	PAGE CHANGED	SER. No.	DOC. PRODUCT.
INI.	2025-09	Saito	-----		NASU

Security Patch Package name

SPZX17-20104_202508PDR_hotfix_win10_LTSC2021_x64_enu

Applicable Products for the Security Patch

PDR-03A Aitella	V2.02JR000
-----------------	------------

Steps to Apply the Security Patch

Please follow the instruction of applying patches defined by each of the product Security Patch Package. For details, please contact the customer service for each system.

Verified Security Patch List

KB	MSRC	explanation(en-us)	URL/File Name	url
KB2538242_x64	MS11-025	Microsoft Visual C++ 2005 SP1 Redistributable Package [version 8.0.61000]	vc redistrib_x64.exe	https://catalog.s.download.windowsupdate.com/msdownload/update/software/secu/2011/06/vcredist_x64_ee916012783024dac67fc606457377932c826f05.exe
KB2538243_x86	MS11-025	Security Update for Microsoft Visual C++ 2008 Service Pack 1 Redistributable Package (KB2538243)	vc redistrib_x86.exe	https://catalog.s.download.windowsupdate.com/msdownload/update/software/secu/2011/05/vcredist_x86_470640aa4bb7db8e69196b5edb0010933569e98d.exe
KB2538243_x64	MS11-025	Security Update for Microsoft Visual C++ 2008 Service Pack 1 Redistributable Package (KB2538243)	vc redistrib_x64.exe	https://catalog.s.download.windowsupdate.com/msdownload/update/software/secu/2011/05/vcredist_x64_a7c83077b8a28d409e36316d2d7321fa0c cdb7e8.exe
KB2565063_x86	MS11-025	Security Update for Microsoft Visual C++ 2010 Service Pack 1 Redistributable Package (KB2565063)	vc redistrib_x86.exe	https://catalog.s.download.windowsupdate.com/msdownload/update/software/secu/2011/07/vcredist_x86_28c54491be70c38c97849c3d8cfbddd0d3c515cb.exe
KB2565063_x64	MS11-025	Security Update for Microsoft Visual C++ 2010 Service Pack 1 Redistributable Package (KB2565063)	vc redistrib_x64.exe	https://catalog.s.download.windowsupdate.com/msdownload/update/software/secu/2011/07/vcredist_x64_15d032d669078aa6f0f7fd1cbf4115a070bd034d.exe
VS2012_x86	n/a	Microsoft Visual C++ Visual Studio 2012	vc redistrib_x86.exe	https://download.microsoft.com/download/1/6/B/16B06F60-3B20-4FF2-B699-5E9B7962F9AE/VSU3/vcredist_x86.exe
VS2012_x64	n/a	Microsoft Visual C++ Visual Studio 2012	vc redistrib_x64.exe	https://download.microsoft.com/download/1/6/B/16B06F60-3B20-4FF2-B699-5E9B7962F9AE/VSU3/vcredist_x64.exe

KB	MSRC	explanation(en-us)	URL/File Name	url
VS2013_x64	n/a	Visual C++ Redistributable Packages for Visual Studio 2013	vcredist_x64.exe	https://download.microsoft.com/download/2/E/6/2E61CFA4-993B-4DD4-91DA-3737CD5CD6E3/vcredist_x64.exe
VS2022_x86	n/a	Microsoft Visual C++ Visual Studio 2015-2022	VC_redist.x86.exe	https://download.visualstudio.microsoft.com/download/pr/eaab1f82-787d-4fd7-8c73-f782341a0c63/5365A927487945ECB040E143EA770ADBB296074ECE4021B1D14213BDE538C490/VC_redist.x86.exe
VS2022_x64	n/a	Microsoft Visual C++ Visual Studio 2015-2022	VC_redist.x64.exe	https://download.visualstudio.microsoft.com/download/pr/eaab1f82-787d-4fd7-8c73-f782341a0c63/917C37D816488545B70AFFD77D6E486E4DD27E2ECE63F6BBAAF486B178B2B888/VC_redist.x64.exe
KB5031539	n/a	2023-10 Servicing Stack Update for Windows 10 Version 21H2 for x64-based Systems (KB5031539)	ssu-19041.3562-x64_de23c91f483b2e609cec3e4a995639d13205f867.msu	https://catalog.s.download.windowsupdate.com/c/msdownload/update/software/secu/2023/10/ssu-19041.3562-x64_de23c91f483b2e609cec3e4a995639d13205f867.msu
KB5066188	n/a	2025-08 Cumulative Update for Windows 10 Version 21H2 for x64-based Systems (KB5066188)	windows10.0-kb5066188-x64_441ad3d5d5e632d0e649d59e5a3e085ee79200b1.msu	https://catalog.s.download.windowsupdate.com/c/msdownload/update/software/updt/2025/08/windows10.0-kb5066188-x64_441ad3d5d5e632d0e649d59e5a3e085ee79200b1.msu
KB5056577	n/a	2025-07 Cumulative Update for .NET Framework 3.5 and 4.8 for Windows 10 Version 21H2 for x64 (KB5056577)	windows10.0-kb5056577-x64-ndp48_c9ed1bb4f9ed43fc65d4e46d15b1e9230f9a7582.msu	https://catalog.s.download.windowsupdate.com/c/msdownload/update/software/updt/2025/03/windows10.0-kb5056577-x64-ndp48_c9ed1bb4f9ed43fc65d4e46d15b1e9230f9a7582.msu
Edge_x64_139.0.3405.119	n/a	Microsoft Edge-Stable Channel Version 139 Update for x64 based Editions (Build 139.0.3405.119)	microsoftedgeenterprise_x64.cab	https://catalog.s.download.windowsupdate.com/d/msdownload/update/software/updt/2025/08/microsoftedgeenterprise_x64_03ecc7d78ab0cdcc698a13ebd468d20215e19b9c.cab
KB890830_v5.135	n/a	Windows Malicious Software Removal Tool x64 - v5.135 (KB890830)	windows-kb890830-x64-v5.135_72c48060c5b80784333da5bef85e6d52fddb4b66.exe	https://catalog.s.download.windowsupdate.com/c/msdownload/update/software/updt/2025/08/windows-kb890830-x64-v5.135_72c48060c5b80784333da5bef85e6d52fddb4b66.exe



LEGAL MANUFACTURER

CANON MEDICAL SYSTEMS CORPORATION

1385, SHIMOISHIGAMI, OTAWARA-SHI, TOCHIGI 324-8550, JAPAN