

**PRODUCT SECURITY PATCH LIST  
FOR  
NUCLEAR MEDICINE EQUIPMENT  
(2G984-0116EN)**

**CANON MEDICAL SYSTEMS CORPORATION**

## Trademarks

Windows® is a registered trademark of Microsoft Corporation in the United States and other countries.

This document may also include trademarks or registered trademarks of other companies.

Note that the trademark symbol "™" and the registered trademark symbol "®" may or may not be used in this document.

### **IMPORTANT!**

1. No part of this document may be copied or reprinted, in whole or in part, without written permission.
2. The contents of this document are subject to change without prior notice and without legal obligation.
3. The contents of this document are correct to the best of our knowledge. Please inform us of any ambiguous or erroneous descriptions, missing information, etc.

| REVISION RECORD |                   |                   |                 |             |                  |
|-----------------|-------------------|-------------------|-----------------|-------------|------------------|
| REV.            | DATE<br>(YYYY-MM) | REASON<br>/AUTHOR | PAGE<br>CHANGED | SER.<br>No. | DOC.<br>PRODUCT. |
| INI.            | 2025-12           | Shishido          | -----           |             | NASU             |

Security Patch Package name

SFDX60-07067\_202511NM\_hotfix\_win10\_LTSC2021\_x64\_enu\_AcqWS

Applicable Products for the Security Patch

|           |              |
|-----------|--------------|
| GCA-9300R | V5.0 SP0001J |
|-----------|--------------|

Steps to Apply the Security Patch

Please follow the instruction of applying patches defined by each of the product Security Patch Package. For details, please contact the customer service for each system.

Verified Security Patch List

| KB            | MSRC     | explanation(en-us)                                                                           | URL/File Name                                                   | url                                                                                                                                                                                                                                                                                                                                                                           |
|---------------|----------|----------------------------------------------------------------------------------------------|-----------------------------------------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| KB2565063_x86 | MS11-025 | Microsoft Visual C++ 2010 Service Pack 1 Redistributable Package MFC Security Update         | vc_redist_x86.exe                                               | <a href="https://catalog.s.download.windowsupdate.com/msdownload/update/software/secu/2011/07/vcredist_x86_28c54491be70c38c97849c3d8cfbddd0d3c515cb.exe">https://catalog.s.download.windowsupdate.com/msdownload/update/software/secu/2011/07/vcredist_x86_28c54491be70c38c97849c3d8cfbddd0d3c515cb.exe</a>                                                                   |
| KB2565063_x64 | MS11-025 | Microsoft Visual C++ 2010 Service Pack 1 Redistributable Package MFC Security Update         | vc_redist_x64.exe                                               | <a href="https://catalog.s.download.windowsupdate.com/msdownload/update/software/secu/2011/07/vcredist_x64_15d032d669078aa6f0f7fd1cbf4115a070bd034d.exe">https://catalog.s.download.windowsupdate.com/msdownload/update/software/secu/2011/07/vcredist_x64_15d032d669078aa6f0f7fd1cbf4115a070bd034d.exe</a>                                                                   |
| VS2012_x86    | n/a      | Visual C++ Redistributable for Visual Studio 2012 Update 4                                   | vc_redist_x86.exe                                               | <a href="https://download.microsoft.com/download/1/6/B/16B06F60-3B20-4FF2-B699-5E9B7962F9AE/VSU_4/vcredist_x86.exe">https://download.microsoft.com/download/1/6/B/16B06F60-3B20-4FF2-B699-5E9B7962F9AE/VSU_4/vcredist_x86.exe</a>                                                                                                                                             |
| VS2012_x64    | n/a      | Visual C++ Redistributable for Visual Studio 2012 Update 4                                   | vc_redist_x64.exe                                               | <a href="https://download.microsoft.com/download/1/6/B/16B06F60-3B20-4FF2-B699-5E9B7962F9AE/VSU_4/vcredist_x64.exe">https://download.microsoft.com/download/1/6/B/16B06F60-3B20-4FF2-B699-5E9B7962F9AE/VSU_4/vcredist_x64.exe</a>                                                                                                                                             |
| VS2017_x64    | n/a      | Microsoft Visual C++ Visual Studio 2017                                                      | vc_redist.x64.exe                                               | <a href="https://download.visualstudio.microsoft.com/download/pr/4100b84d-1b4d-487d-9f89-1354a7138c8f/5B0CBB977F2F5253B1EBE5C9D30EDBDA35DBD68FB70DE7AF5FAAC6423DB575B5/VC_redist.x64.exe">https://download.visualstudio.microsoft.com/download/pr/4100b84d-1b4d-487d-9f89-1354a7138c8f/5B0CBB977F2F5253B1EBE5C9D30EDBDA35DBD68FB70DE7AF5FAAC6423DB575B5/VC_redist.x64.exe</a> |
| KB5031539     | n/a      | 2023-10 Servicing Stack Update for Windows 10 Version 21H2 for x64-based Systems (KB5031539) | ssu-19041.3562-x64_de23c91f483b2e609cec3e4a995639d13205f867.msu | <a href="https://catalog.s.download.windowsupdate.com/c/msdownload/update/software/secu/2023/10/ssu-19041.3562-x64_de23c91f483b2e609cec3e4a995639d13205f867.msu">https://catalog.s.download.windowsupdate.com/c/msdownload/update/software/secu/2023/10/ssu-19041.3562-x64_de23c91f483b2e609cec3e4a995639d13205f867.msu</a>                                                   |

| KB              | MSRC | explanation(en-us)                                                                                       | URL/File Name                                                                | url                                                                                                                                                                                                                                                                                                                                                   |
|-----------------|------|----------------------------------------------------------------------------------------------------------|------------------------------------------------------------------------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| KB5068781       | n/a  | 2025-11 Cumulative Update for Windows 10 Version 21H2 for x64-based Systems (KB5068781)                  | windows10.0-kb5068781-x64_33b4e6ee5956ae0f4be6a37716b0ef4fd1720e05.msu       | <a href="https://catalog.s.download.windowsupdate.com/c/msdownload/update/software/secu/2025/11/windows10.0-kb5068781-x64_33b4e6ee5956ae0f4be6a37716b0ef4fd1720e05.msu">https://catalog.s.download.windowsupdate.com/c/msdownload/update/software/secu/2025/11/windows10.0-kb5068781-x64_33b4e6ee5956ae0f4be6a37716b0ef4fd1720e05.msu</a>             |
| KB5066135       | n/a  | 2025-10 Cumulative Update for .NET Framework 3.5 and 4.8 for Windows 10 Version 21H2 for x64 (KB5066135) | windows10.0-kb5066135-x64-ndp48_0838b78fa58e8f54c56ed6bc4fbfc762b829a87f.msu | <a href="https://catalog.s.download.windowsupdate.com/c/msdownload/update/software/secu/2025/09/windows10.0-kb5066135-x64-ndp48_0838b78fa58e8f54c56ed6bc4fbfc762b829a87f.msu">https://catalog.s.download.windowsupdate.com/c/msdownload/update/software/secu/2025/09/windows10.0-kb5066135-x64-ndp48_0838b78fa58e8f54c56ed6bc4fbfc762b829a87f.msu</a> |
| KB890830_v5.137 | n/a  | Windows Malicious Software Removal Tool x64 - v5.137 (KB890830)                                          | windows-kb890830-x64-v5.137.exe                                              | <a href="https://catalog.s.download.windowsupdate.com/c/msdownload/update/software/uprl/2025/11/windows-kb890830-x64-v5.137_d26124d4bdd1a7259db3b081f198db147c7dec0b.exe">https://catalog.s.download.windowsupdate.com/c/msdownload/update/software/uprl/2025/11/windows-kb890830-x64-v5.137_d26124d4bdd1a7259db3b081f198db147c7dec0b.exe</a>         |



LEGAL MANUFACTURER

**CANON MEDICAL SYSTEMS CORPORATION**

---

1385, SHIMOISHIGAMI, OTAWARA-SHI, TOCHIGI 324-8550, JAPAN