

**PRODUCT SECURITY PATCH LIST
FOR
DIAGNOSTIC ULTRASOUND SYSTEM
(2G984-0147EN)**

Canon Inc.

Trademarks

Windows® is a registered trademark of Microsoft Corporation in the United States and other countries.

This document may also include trademarks or registered trademarks of other companies.

Note that the trademark symbol "™" and the registered trademark symbol "®" may or may not be used in this document.

IMPORTANT!

1. No part of this document may be copied or reprinted, in whole or in part, without written permission.
2. The contents of this document are subject to change without prior notice and without legal obligation.
3. The contents of this document are correct to the best of our knowledge. Please inform us of any ambiguous or erroneous descriptions, missing information, etc.

REVISION RECORD						
REV.	DATE (YYYY-MM)	REASON	/AUTHOR	PAGE CHANGED	SER. No.	DOC. PRODUCT.
INI.	2026-04		Murakami	-----		BU

Security Patch Package name

SPZM37-20137A_202603UL_hotfix_win11_LTSC2024_x64_enu

Applicable Products for the Security Patch

TUS-AI900/800/700 Without UITB-AI900A option. Standard whitelisting enabled environment is not applicable.	V9.0 SP0000* - V9.0 SP0005*
---	--------------------------------

Steps to Apply the Security Patch

Please follow the instruction of applying patches defined by each of the product Security Patch Package. For details, please contact the customer service for each system.

Verified Security Patch List

KB	MSRC	explanation(en-us)	URL/File Name	url
KB2565063_x64	MS11-025	Security Update for Microsoft Visual C++ 2010 Service Pack 1 Redistributable Package (KB2565063)	vcredist_x64.exe	https://catalog.s.download.windowsupdate.com/msdownload/update/software/secul/2011/07/vcredist_x64_15d032d669078aa6f0f7fd1cbf4115a070bd034d.exe
KB2565063_x86	MS11-025	Security Update for Microsoft Visual C++ 2010 Service Pack 1 Redistributable Package (KB2565063)	vcredist_x86.exe	https://catalog.s.download.windowsupdate.com/msdownload/update/software/secul/2011/07/vcredist_x86_28c54491be70c38c97849c3d8cfbffd0d3c515cb.exe
VS2012_x86	n/a	Visual C++ Redistributable for Visual Studio 2012 Update 4	vcredist_x86.exe	https://download.microsoft.com/download/1/6/B/16B06F60-3B20-4FF2-B699-5E9B7962F9AE/VSU_4/vcredist_x86.exe
VS2012_x64	n/a	Visual C++ Redistributable for Visual Studio 2012 Update 4	vcredist_x64.exe	https://download.microsoft.com/download/1/6/B/16B06F60-3B20-4FF2-B699-5E9B7962F9AE/VSU_4/vcredist_x64.exe
VS2013_x64	n/a	Microsoft Visual C++ 2013 Update 5 Redistributable Package	vcredist_x64.exe	https://download.microsoft.com/download/c/c/2/cc2df5f8-4454-44b4-802d-5ea68d086676/vcredist_x64.exe
VS2013_x86	n/a	Microsoft Visual C++ 2013 Update 5 Redistributable Package	vcredist_x86.exe	https://download.microsoft.com/download/c/c/2/cc2df5f8-4454-44b4-802d-5ea68d086676/vcredist_x86.exe

KB	MSRC	explanation(en-us)	URL/File Name	url
VS2019_x86	n/a	Microsoft Visual C++ 2015-2019 Redistributable (x86)	vc_redist.x86.exe	https://download.visualstudio.microsoft.com/download/pr/73b58d04-0049-47d1-9f54-1784792c71cd/80C7969F4E05002A0CD820B746E0ACB7406D4B85E52EF096707315B390927824/VC_redist.x86.exe
VS2019_x64	n/a	Microsoft Visual C++ 2015-2019 Redistributable (x64)	vc_redist.x64.exe	https://download.visualstudio.microsoft.com/download/pr/cc0046d4-e7b4-45a1-bd46-b1c079191224/9c4042a4c2e6d1f661f4c58cf4d129e9/vc_redist.x64.exe
Edge_x64_145.0.3800.97	n/a	Microsoft Edge-Stable Channel Version 145 Update for x64 based Editions (Build 145.0.3800.97)	microsoftedgeenterprisex64.cab	https://catalog.s.download.windowsupdate.com/d/msdownload/update/software/updt/2026/03/microsoftedgeenterprisex64_6057eab9dbe52634959c3b24840c476d2e521b23.cab
KB5043080	n/a	2024-09 Cumulative Update for Windows 11 Version 24H2 for x64-based Systems (KB5043080)	windows11.0-kb5043080-x64_953449672073f8fb99badb4cc6d5d7849b9c83e8.msu	https://catalog.sf.dl.delivery.mp.microsoft.com/filestreamingservice/files/d8b7f92b-bd35-4b4c-96e5-46ce984b31e0/public/windows11.0-kb5043080-x64_953449672073f8fb99badb4cc6d5d7849b9c83e8.msu
KB4052623_4.18.25100.9008	n/a	Update for Microsoft Defender Antivirus antimalware platform - KB4052623 (Version 4.18.25100.9008) - Current Channel (Broad)	updateplatform.exe	https://catalog.s.download.windowsupdate.com/d/msdownload/update/software/defu/2025/11/updateplatform.amd64fre_bf6fa365cd96d7e97da42a33f63a6fe5579db805.exe
KB5079473	n/a	2026-03 Cumulative Update for Windows 11 Version 24H2 for x64-based Systems (KB5079473) (26100.8037)	windows11.0-kb5079473-x64_651d0cdb7665e03bb80b4c198784b255081032d2.msu	https://catalog.sf.dl.delivery.mp.microsoft.com/filestreamingservice/files/db81d709-c552-47ed-a3b9-ba77dfff9972/public/windows11.0-kb5079473-x64_651d0cdb7665e03bb80b4c198784b255081032d2.msu
KB5066131	n/a	2025-10 Cumulative Update for .NET Framework 3.5 and 4.8.1 for Windows 11, version 24H2 for x64 (KB5066131)	windows11.0-kb5066131-x64-ndp481_b2ab1290d276d5cb9c9d03dce5ad2d6e2b66f615.msu	https://catalog.sf.dl.delivery.mp.microsoft.com/filestreamingservice/files/035cd7e9-5769-4168-80e4-d73bc6b746c8/public/windows11.0-kb5066131-x64-ndp481_b2ab1290d276d5cb9c9d03dce5ad2d6e2b66f615.msu
KB890830_v5.139	n/a	Windows Malicious Software Removal Tool x64 - v5.139 (KB890830)	windows-kb890830-x64-v5.139.exe	https://catalog.s.download.windowsupdate.com/d/msdownload/update/software/uprl/2026/02/windows-kb890830-x64-v5.139_c6a3e646b705898985bbd3b88ab2f672b84a8784.exe



LEGAL MANUFACTURER

Canon Inc.

30-2, SHIMOMARUKO 3-CHOME, OHTA-KU, TOKYO 146-8501, JAPAN