

**PRODUCT SECURITY PATCH LIST  
FOR  
NUCLEAR MEDICINE EQUIPMENT  
(2G984-0150EN)**

Canon Inc.

## Trademarks

Windows® is a registered trademark of Microsoft Corporation in the United States and other countries.

This document may also include trademarks or registered trademarks of other companies.

Note that the trademark symbol "™" and the registered trademark symbol "®" may or may not be used in this document.

### **IMPORTANT!**

1. No part of this document may be copied or reprinted, in whole or in part, without written permission.
2. The contents of this document are subject to change without prior notice and without legal obligation.
3. The contents of this document are correct to the best of our knowledge. Please inform us of any ambiguous or erroneous descriptions, missing information, etc.

<b>REVISION RECORD</b>						
REV.	DATE (YYYY-MM)	REASON	/AUTHOR	PAGE CHANGED	SER. No.	DOC. PRODUCT.
INI.	2026-03		Soeta	-----		BU

Security Patch Package name

SFDX60-07069\_202603NM\_hotfix\_win10\_AcqWS\_LTSB2016\_x64\_enu

Applicable Products for the Security Patch

GCA-9300R	V4.1 SP0000J
-----------	--------------

Steps to Apply the Security Patch

Please follow the instruction of applying patches defined by each of the product Security Patch Package. For details, please contact the customer service for each system.

Verified Security Patch List

KB	MSRC	explanation(en-us)	URL/File Name	url
KB2565063_x86	MS11-025	Microsoft Visual C++ 2010 Service Pack 1 Redistributable Package MFC Security Update	vc redistrib_x86.exe	<a href="https://catalog.s.download.windowsupdate.com/msdownload/update/software/secu/2011/07/vcredist_x86_28c54491be70c38c97849c3d8cfbddd0d3c515cb.exe">https://catalog.s.download.windowsupdate.com/msdownload/update/software/secu/2011/07/vcredist_x86_28c54491be70c38c97849c3d8cfbddd0d3c515cb.exe</a>
KB2565063_x64	MS11-025	Microsoft Visual C++ 2010 Service Pack 1 Redistributable Package MFC Security Update	vc redistrib_x64.exe	<a href="https://catalog.s.download.windowsupdate.com/msdownload/update/software/secu/2011/07/vcredist_x64_15d032d669078aa6f0f7fd1cbf4115a070bd034d.exe">https://catalog.s.download.windowsupdate.com/msdownload/update/software/secu/2011/07/vcredist_x64_15d032d669078aa6f0f7fd1cbf4115a070bd034d.exe</a>
VS2012_x86	n/a	Visual C++ Redistributable for Visual Studio 2012 Update 4	vc redistrib_x86.exe	<a href="https://download.microsoft.com/download/1/6/B/16B06F60-3B20-4FF2-B699-5E9B7962F9AE/VSU_4/vcredist_x86.exe">https://download.microsoft.com/download/1/6/B/16B06F60-3B20-4FF2-B699-5E9B7962F9AE/VSU_4/vcredist_x86.exe</a>
VS2012_x64	n/a	Visual C++ Redistributable for Visual Studio 2012 Update 4	vc redistrib_x64.exe	<a href="https://download.microsoft.com/download/1/6/B/16B06F60-3B20-4FF2-B699-5E9B7962F9AE/VSU_4/vcredist_x64.exe">https://download.microsoft.com/download/1/6/B/16B06F60-3B20-4FF2-B699-5E9B7962F9AE/VSU_4/vcredist_x64.exe</a>
VS2017_x64	n/a	Microsoft Visual C++ Visual Studio 2017	VC_redist.x64.exe	<a href="https://download.visualstudio.microsoft.com/download/pr/12328699/a80e967515b2a4faf37bf15387f1c5c3/VC_redist.x64.exe">https://download.visualstudio.microsoft.com/download/pr/12328699/a80e967515b2a4faf37bf15387f1c5c3/VC_redist.x64.exe</a>
KB5075902	n/a	2026-02 Servicing Stack Update for Windows 10 Version 1607 for x64-based Systems (KB5075902)	windows10.0-kb5075902-x64_a840f48f81e79f45131128f9f823633a59f57879.msu	<a href="https://catalog.s.download.windowsupdate.com/d/msdownload/update/software/secu/2026/02/windows10.0-kb5075902-x64_a840f48f81e79f45131128f9f823633a59f57879.msu">https://catalog.s.download.windowsupdate.com/d/msdownload/update/software/secu/2026/02/windows10.0-kb5075902-x64_a840f48f81e79f45131128f9f823633a59f57879.msu</a>

KB	MSRC	explanation(en-us)	URL/File Name	url
KB5078938	n/a	2026-03 Cumulative Update for Windows 10 Version 1607 for x64-based Systems (KB5078938)	windows10.0-kb5078938-x64_4f2644d71dfb3459b6777dfb1a3a4e16cf3877f5.msu	<a href="https://catalog.s.download.windowsupdate.com/c/msdownload/update/software/secu/2026/03/windows10.0-kb5078938-x64_4f2644d71dfb3459b6777dfb1a3a4e16cf3877f5.msu">https://catalog.s.download.windowsupdate.com/c/msdownload/update/software/secu/2026/03/windows10.0-kb5078938-x64_4f2644d71dfb3459b6777dfb1a3a4e16cf3877f5.msu</a>
KB890830_v5.139	n/a	Windows Malicious Software Removal Tool x64 - v5.139 (KB890830)	windows-kb890830-x64-v5.139_c6a3e646b705898985bbd3b88ab2f672b84a8784.exe	<a href="https://catalog.s.download.windowsupdate.com/d/msdownload/update/software/upri/2026/02/windows-kb890830-x64-v5.139_c6a3e646b705898985bbd3b88ab2f672b84a8784.exe">https://catalog.s.download.windowsupdate.com/d/msdownload/update/software/upri/2026/02/windows-kb890830-x64-v5.139_c6a3e646b705898985bbd3b88ab2f672b84a8784.exe</a>



LEGAL MANUFACTURER

**Canon Inc.**

---

30-2, SHIMOMARUKO 3-CHOME, OHTA-KU, TOKYO 146-8501, JAPAN