

**PRODUCT SECURITY PATCH LIST
FOR
X-RAY CT SCANNER
(2G984-0156EN)**

Canon Inc.

Trademarks

Windows® is a registered trademark of Microsoft Corporation in the United States and other countries.

This document may also include trademarks or registered trademarks of other companies.

Note that the trademark symbol "™" and the registered trademark symbol "®" may or may not be used in this document.

IMPORTANT!

1. No part of this document may be copied or reprinted, in whole or in part, without written permission.
2. The contents of this document are subject to change without prior notice and without legal obligation.
3. The contents of this document are correct to the best of our knowledge. Please inform us of any ambiguous or erroneous descriptions, missing information, etc.

| REVISION RECORD | | | | | | |
|------------------------|-------------------|--------|----------|-----------------|-------------|------------------|
| REV. | DATE (YYYY-MM) | REASON | /AUTHOR | PAGE CHANGED | SER. No. | DOC. PRODUCT. |
| INI. | 2026-04 | | Shishido | ----- | | BU |

Security Patch Package name

SPZX76-20102_202603CT_hotfix_win10_LTSC2019_x64_enu

Applicable Products for the Security Patch

| | |
|--|-----------------------|
| Aquilion Prime SP V10.2, V10.4, V10.17, Base V16.1SP0001 is not applicable | TSX-303B |
| Aquilion Lightning V10.2, V10.4, V10.17, Base V16.1SP0001 is not applicable | TSX-036A |
| Aquilion ONE V10.0, V10.4, V10.11, V10.12, V10.16 | TSX-305A/ TSX-306A |
| Aquilion Exceed LB V10.9, V10.18 | TSX-202A |
| Aquilion LB V6.6, V6.7 | TSX-201A |
| Aquilion Lightning V10.5, V10.20 | TSX-035A |
| Aquilion Start V10.5, V10.20 | TSX-037A |
| Aquilion Precision V10.10, V10.14, V10.19 | TSX-304A |
| Aquilion Serve V1.3, V1.5, V1.7, V2.0 | TSX-307A |
| Aquilion Serve SP V1.3, V1.5, V1.7, V2.0 iBase V2.0SP0001 is not applicable | TSX-307B |
| Aquilion ONE V1.5, V1.7, V2.0 iBASE V1.1 SP0102* or later | TSX-306A/ TSX-308A |

Steps to Apply the Security Patch

Please follow the instruction of applying patches defined by each of the product Security Patch Package. For details, please contact the customer service for each system.

Verified Security Patch List

| KB | MSRC | explanation(en-us) | URL/File Name | url |
|---------------|----------|--|----------------------|---|
| KB2538242_x86 | MS11-025 | Microsoft Visual C++ 2005 Service Pack 1 Redistributable Package MFC Security Update | vc redistrib_x86.exe | https://catalog.s.download.windowsupdate.com/msdownload/update/software/secu/2011/06/vcredist_x86_b8fab0bb7f62a24ddfe77b19cd9a1451abd7b847.exe |
| KB2538242_x64 | MS11-025 | Microsoft Visual C++ 2005 Service Pack 1 Redistributable Package MFC Security Update | vc redistrib_x64.exe | https://catalog.s.download.windowsupdate.com/msdownload/update/software/secu/2011/06/vcredist_x64_ee916012783024dac67fc606457377932c826f05.exe |

| KB | MSRC | explanation(en-us) | URL/File Name | url |
|-----------------|----------|--|--|---|
| KB2538243_x86 | MS11-025 | Security Update for Microsoft Visual C++ 2008 Service Pack 1 Redistributable Package (KB2538243) | vcredist_x86.exe | https://catalog.s.download.windowsupdate.com/msdownload/update/software/secu/2011/05/vcredist_x86_470640aa4bb7db8e69196b5edb0010933569e98d.exe |
| KB2538243_x64 | MS11-025 | Security Update for Microsoft Visual C++ 2008 Service Pack 1 Redistributable Package (KB2538243) | vcredist_x64.exe | https://catalog.s.download.windowsupdate.com/msdownload/update/software/secu/2011/05/vcredist_x64_a7c83077b8a28d409e36316d2d7321fa0ccdb7e8.exe |
| KB2565063_x86 | MS11-025 | Microsoft Visual C++ 2010 Service Pack 1 Redistributable Package MFC Security Update | vcredist_x86.exe | https://catalog.s.download.windowsupdate.com/msdownload/update/software/secu/2011/07/vcredist_x86_28c54491be70c38c97849c3d8cbfdd0d3c515cb.exe |
| KB2565063_x64 | MS11-025 | Microsoft Visual C++ 2010 Service Pack 1 Redistributable Package MFC Security Update | vcredist_x64.exe | https://catalog.s.download.windowsupdate.com/msdownload/update/software/secu/2011/07/vcredist_x64_15d032d669078aa6f0f7fd1cbf4115a070bd034d.exe |
| VS2012_x86 | n/a | Visual C++ Redistributable for Visual Studio 2012 Update 4 | vcredist_x86.exe | https://download.microsoft.com/download/1/6/B/16B06F60-3B20-4FF2-B699-5E9B7962F9AE/VSU_4/vcredist_x86.exe |
| VS2012_x64 | n/a | Visual C++ Redistributable for Visual Studio 2012 Update 4 | vcredist_x64.exe | https://download.microsoft.com/download/1/6/B/16B06F60-3B20-4FF2-B699-5E9B7962F9AE/VSU_4/vcredist_x64.exe |
| VS2013_x86 | n/a | Microsoft Visual C++ 2013 Update 5 Redistributable Package | vcredist_x86.exe | https://download.microsoft.com/download/c/c/2/cc2df5f8-4454-44b4-802d-5ea68d086676/vcredist_x86.exe |
| VS2013_x64 | n/a | Microsoft Visual C++ 2013 Update 5 Redistributable Package | vcredist_x64.exe | https://download.microsoft.com/download/c/c/2/cc2df5f8-4454-44b4-802d-5ea68d086676/vcredist_x64.exe |
| KB5005112 | n/a | 2021-08 Servicing Stack Update for Windows 10 Version 1809 for x64-based Systems (KB5005112) | windows10.0-kb5005112-x64_81d09dc6978520e1a6d44b3b15567667f83eba2c.msu | https://catalog.s.download.windowsupdate.com/d/msdownload/update/software/secu/2021/08/windows10.0-kb5005112-x64_81d09dc6978520e1a6d44b3b15567667f83eba2c.msu |
| KB5078752 | n/a | 2026-03 Cumulative Update for Windows 10 Version 1809 for x64-based Systems (KB5078752) | windows10.0-kb5078752-x64_d19dce98bd2132ac8dcf4ae644ff36ec1ac7a734.msu | https://catalog.s.download.windowsupdate.com/c/msdownload/update/software/secu/2026/03/windows10.0-kb5078752-x64_d19dce98bd2132ac8dcf4ae644ff36ec1ac7a734.msu |
| KB890830_v5.139 | n/a | Windows Malicious Software Removal Tool x64 - v5.139 (KB890830) | windows-kb890830-x64-v5.139.exe | https://catalog.s.download.windowsupdate.com/d/msdownload/update/software/uprl/2026/02/windows-kb890830-x64-v5.139_c6a3e646b705898985bbd3b88ab2f672b84a8784.exe |
| KB5066143 | n/a | 2025-10 Cumulative Update for .NET Framework 3.5 and 4.7.2 for Windows 10 Version 1809 for x64 (KB5066143) | windows10.0-kb5066143-x64_461230ac1884e9691f54936b91ff04e46b80c2bc.msu | https://catalog.s.download.windowsupdate.com/d/msdownload/update/software/secu/2025/09/windows10.0-kb5066143-x64_461230ac1884e9691f54936b91ff04e46b80c2bc.msu |

| KB | MSRC | explanation(en-us) | URL/File Name | url |
|-----------|------|--|--|---|
| KB5019181 | n/a | 2023-03 Update for Windows 10 Version 1809 for x64-based Systems (KB5019181) | windows10.0-kb5019181-x64_01173052877dc5762f73069a958c108afd1125be.msu | https://catalog.s.download.windowsupdate.com/d/msdownload/update/software/updt/2023/02/windows10.0-kb5019181-x64_01173052877dc5762f73069a958c108afd1125be.msu |



LEGAL MANUFACTURER

Canon Inc.

30-2, SHIMOMARUKO 3-CHOME, OHTA-KU, TOKYO 146-8501, JAPAN