

**PRODUCT SECURITY PATCH LIST
FOR
DIGITAL RADIOGRAPHY SYSTEMS
(2G984-0159EN)**

Canon Inc.

Trademarks

Windows® is a registered trademark of Microsoft Corporation in the United States and other countries.

This document may also include trademarks or registered trademarks of other companies.

Note that the trademark symbol "™" and the registered trademark symbol "®" may or may not be used in this document.

IMPORTANT!

1. No part of this document may be copied or reprinted, in whole or in part, without written permission.
2. The contents of this document are subject to change without prior notice and without legal obligation.
3. The contents of this document are correct to the best of our knowledge. Please inform us of any ambiguous or erroneous descriptions, missing information, etc.

REVISION RECORD					
REV.	DATE (YYYY-MM)	REASON	PAGE CHANGED	SER. No.	DOC. PRODUCT.
INI.	2026-04		Ishii	-----	BU

Security Patch Package name

SPZX17-20119_202603HDR_EREf_hotfix_win10_LTSC2021_x64_enu

Applicable Products for the Security Patch

XDRK-EREf	V2.00*R001 or later
-----------	---------------------

Steps to Apply the Security Patch

Please follow the instruction of applying patches defined by each of the product Security Patch Package. For details, please contact the customer service for each system.

Verified Security Patch List

KB	MSRC	explanation(en-us)	URL/File Name	url
KB2565063_x64	MS11-025	Microsoft Visual C++ 2010 Service Pack 1 Redistributable Package MFC Security Update	vc_redist_x64.exe	https://catalog.s.download.windowsupdate.com/msdownload/update/software/secu/2011/07/vcredist_x64_15d032d669078aa6f0f7fd1cbf4115a070bd034d.exe
VS2019_x86_14.29.30135.0	n/a	Microsoft Visual C++ 2015-2019 Redistributable (x86)	vc_redist.x86.exe	https://download.visualstudio.microsoft.com/download/pr/73b58d04-0049-47d1-9f54-1784792c71cd/80C7969F4E05002A0CD820B746E0ACB7406D4B85E52EF096707315B390927824/VC_redist.x86.exe
VS2019_x64_14.29.30135.0	n/a	Microsoft Visual C++ 2015-2019 Redistributable (x64)	vc_redist.x64.exe	https://download.visualstudio.microsoft.com/download/pr/e332b62b-e04e-4cbd-8c3b-41bc13761c41/9B9DD72C27AB1DB081DE56BB7B73BEE9A00F60D14ED8E6FDE45DAB3E619B5F04/VC_redist.x64.exe
KB5031539	n/a	2023-10 Servicing Stack Update for Windows 10 Version 21H2 for x64-based Systems (KB5031539)	ssu-19041.3562-x64_de23c91f483b2e609cec3e4a995639d13205f867.msu	https://catalog.s.download.windowsupdate.com/c/msdownload/update/software/secu/2023/10/ssu-19041.3562-x64_de23c91f483b2e609cec3e4a995639d13205f867.msu
KB890830_v5.139	n/a	Windows Malicious Software Removal Tool x64 - v5.139 (KB890830)	windows-kb890830-x64-v5.139.exe	https://catalog.s.download.windowsupdate.com/d/msdownload/update/software/uprl/2026/02/windows-kb890830-x64-v5.139_c6a3e646b705898985bbd3b88ab2f672b84a8784.exe

KB	MSRC	explanation(en-us)	URL/File Name	url
KB5078885	n/a	2026-03 Cumulative Update for Windows 10 Version 21H2 for x64-based Systems (KB5078885)	windows10.0-kb5078885-x64_8013483b567f16e057931c30725c6c8723007a31.msu	https://catalog.s.download.windowsupdate.com/d/msdownload/update/software/secu/2026/03/windows10.0-kb5078885-x64_8013483b567f16e057931c30725c6c8723007a31.msu
KB5066135	n/a	2025-10 Cumulative Update for .NET Framework 3.5 and 4.8 for Windows 10 Version 21H2 for x64 (KB5066135)	windows10.0-kb5066135-x64-ndp48_0838b78fa58e8f54c56ed6bc4fbfc762b829a87f.msu	https://catalog.s.download.windowsupdate.com/c/msdownload/update/software/secu/2025/09/windows10.0-kb5066135-x64-ndp48_0838b78fa58e8f54c56ed6bc4fbfc762b829a87f.msu
KB4052623_4.18.26020.6	n/a	Update for Microsoft Defender Antivirus antimalware platform - KB4052623 (Version 4.18.26020.6) - Current Channel (Broad)	updateplatform.exe	https://catalog.s.download.windowsupdate.com/c/msdownload/update/software/defu/2026/03/updateplatform.amd64fre_a94fdbcbdd69a5cab6032175a4be11f9a52db59f.exe
Edge_x64_146.0.3856.109	n/a	Microsoft Edge-Stable Channel Version 146 Update for x64 based Editions (Build 146.0.3856.109)	microsoftedgeenterprisex64.cab	https://catalog.s.download.windowsupdate.com/d/msdownload/update/software/updt/2026/04/microsoftedgeenterprisex64_03da6dc12ba5999899e743c8fe63aef72b578c5e.cab



LEGAL MANUFACTURER

Canon Inc.

30-2, SHIMOMARUKO 3-CHOME, OHTA-KU, TOKYO 146-8501, JAPAN