

**PRODUCT SECURITY PATCH LIST
FOR
DIGITAL RADIOGRAPHY SYSTEM
(2G984-0167EN)**

CANON INC.

Trademarks

Windows® is a registered trademark of Microsoft Corporation in the United States and other countries.

This document may also include trademarks or registered trademarks of other companies.

Note that the trademark symbol "™" and the registered trademark symbol "®" may or may not be used in this document.

IMPORTANT!

1. No part of this document may be copied or reprinted, in whole or in part, without written permission.
2. The contents of this document are subject to change without prior notice and without legal obligation.
3. The contents of this document are correct to the best of our knowledge. Please inform us of any ambiguous or erroneous descriptions, missing information, etc.

| REVISION RECORD | | | | | | |
|------------------------|-------------------|--------|---------|-----------------|-------------|------------------|
| REV. | DATE (YYYY-MM) | REASON | /AUTHOR | PAGE CHANGED | SER. No. | DOC. PRODUCT. |
| INI. | 2026-05 | | Soeta | ----- | | BU |

Security Patch Package name

SPZX17-20123_202605PDR_hotfix_win10_LTSC2021_x64_enu

Applicable Products for the Security Patch

| | |
|-----------------|-------------------------|
| PDR-03A Aitella | V2.00JR000 - V2.02JR002 |
|-----------------|-------------------------|

Steps to Apply the Security Patch

Please follow the instruction of applying patches defined by each of the product Security Patch Package. For details, please contact the customer service for each system.

Verified Security Patch List

| KB | MSRC | explanation(en-us) | URL/File Name | url |
|---------------|----------|--|----------------------|---|
| KB2538242_x86 | MS11-025 | Microsoft Visual C++ 2005 SP1 Redistributable Package [version 8.0.61001] | vc redistrib_x86.exe | https://catalog.s.download.windowsupdate.com/msdownload/update/software/secu/2011/06/vcredist_x86_b8fab0bb7f62a24ddfe77b19cd9a1451abd7b847.exe |
| KB2538242_x64 | MS11-025 | Microsoft Visual C++ 2005 SP1 Redistributable Package [version 8.0.61000] | vc redistrib_x64.exe | https://catalog.s.download.windowsupdate.com/msdownload/update/software/secu/2011/06/vcredist_x64_ee916012783024dac67fc606457377932c826f05.exe |
| KB2538243_x86 | MS11-025 | Security Update for Microsoft Visual C++ 2008 Service Pack 1 Redistributable Package (KB2538243) | vc redistrib_x86.exe | https://catalog.s.download.windowsupdate.com/msdownload/update/software/secu/2011/05/vcredist_x86_470640aa4bb7db8e69196b5edb0010933569e98d.exe |
| KB2538243_x64 | MS11-025 | Security Update for Microsoft Visual C++ 2008 Service Pack 1 Redistributable Package (KB2538243) | vc redistrib_x64.exe | https://catalog.s.download.windowsupdate.com/msdownload/update/software/secu/2011/05/vcredist_x64_a7c83077b8a28d409e36316d2d7321fa0ccdb7e8.exe |
| KB2565063_x86 | MS11-025 | Security Update for Microsoft Visual C++ 2010 Service Pack 1 Redistributable Package (KB2565063) | vc redistrib_x86.exe | https://catalog.s.download.windowsupdate.com/msdownload/update/software/secu/2011/07/vcredist_x86_28c54491be70c38c97849c3d8cfbfd0d3c515cb.exe |
| KB2565063_x64 | MS11-025 | Security Update for Microsoft Visual C++ 2010 Service Pack 1 Redistributable Package (KB2565063) | vc redistrib_x64.exe | https://catalog.s.download.windowsupdate.com/msdownload/update/software/secu/2011/07/vcredist_x64_15d032d669078aa6f0f7fd1cbf4115a070bd034d.exe |
| VS2012_x86 | n/a | Microsoft Visual C++ Visual Studio 2012 | vc redistrib_x86.exe | https://download.microsoft.com/download/1/6/B/16B06F60-3B20-4FF2-B699-5E9B7962F9AE/VSU3/vcredist_x86.exe |

| KB | MSRC | explanation(en-us) | URL/File Name | url |
|------------------------|------|--|--|---|
| VS2012_x64 | n/a | Microsoft Visual C++ Visual Studio 2012 | vcredist_x64.exe | https://download.microsoft.com/download/1/6/B/16B06F60-3B20-4FF2-B699-5E9B7962F9AE/VSU3/vcredist_x64.exe |
| VS2013_x64 | n/a | Visual C++ Redistributable Packages for Visual Studio 2013 | vcredist_x64.exe | https://download.microsoft.com/download/2/E/6/2E61CFA4-993B-4DD4-91DA-3737CD5CD6E3/vcredist_x64.exe |
| VS2022_x86 | n/a | Microsoft Visual C++ Visual Studio 2015-2022 | VC_redist.x86.exe | https://download.visualstudio.microsoft.com/download/pr/eaab1f82-787d-4fd7-8c73-f782341a0c63/5365A927487945ECB040E143EA770ADBB296074ECE4021B1D14213BDE538C490/VC_redist.x86.exe |
| VS2022_x64 | n/a | Microsoft Visual C++ Visual Studio 2015-2022 | VC_redist.x64.exe | https://download.visualstudio.microsoft.com/download/pr/eaab1f82-787d-4fd7-8c73-f782341a0c63/917C37D816488545B70AFFD77D6E486E4DD27E2ECE63F6BBAAF486B178B2B888/VC_redist.x64.exe |
| KB5031539 | n/a | 2023-10 Servicing Stack Update for Windows 10 Version 21H2 for x64-based Systems (KB5031539) | ssu-19041.3562-x64_de23c91f483b2e609cec3e4a995639d13205f867.msu | https://catalog.s.download.windowsupdate.com/c/msdownload/update/software/secu/2023/10/ssu-19041.3562-x64_de23c91f483b2e609cec3e4a995639d13205f867.msu |
| KB5087544 | n/a | 2026-05 Cumulative Update for Windows 10 Version 21H2 for x64-based Systems (KB5087544) | windows10.0-kb5087544-x64_a3451b8d6af9df06bd43d1b25ec3985fa0db49c3.msu | https://catalog.s.download.windowsupdate.com/d/msdownload/update/software/secu/2026/05/windows10.0-kb5087544-x64_a3451b8d6af9df06bd43d1b25ec3985fa0db49c3.msu |
| KB5087064 | n/a | 2026-05 Cumulative Update for .NET Framework 3.5 and 4.8 for Windows 10 Version 21H2 for x64 (KB5087064) | windows10.0-kb5087064-x64-ndp48_5541701330e7e27d682d1c732d75f9ad27dfb4cd.msu | https://catalog.s.download.windowsupdate.com/d/msdownload/update/software/secu/2026/05/windows10.0-kb5087064-x64-ndp48_5541701330e7e27d682d1c732d75f9ad27dfb4cd.msu |
| Edge_x64_148.0.3967.70 | n/a | Microsoft Edge-Stable Channel Version 148 Update for x64 based Editions (Build 148.0.3967.70) | microsoftedgeenterprise_x64.cab | https://catalog.s.download.windowsupdate.com/c/msdownload/update/software/updt/2026/05/microsoftedgeenterprise_x64_6c55bc80a502776037ef3e818685f64f313c468f.cab |
| KB890830_v5.141 | n/a | Windows Malicious Software Removal Tool x64 - v5.141 (KB890830) | windows-kb890830-x64-v5.141.exe | https://catalog.s.download.windowsupdate.com/c/msdownload/update/software/uprl/2026/05/windows-kb890830-x64-v5.141_28e375ae266a48518b85561b124c6d21b82d84a6.exe |



LEGAL MANUFACTURER

CANON INC.

30-2, SHIMOMARUKO 3-CHOME, OHTA-KU, TOKYO 146-8501, JAPAN