

**PRODUCT SECURITY PATCH LIST
FOR
MRI SCANNER
(2G984-0178EN)**

CANON INC.

Trademarks

Windows® is a registered trademark of Microsoft Corporation in the United States and other countries.

This document may also include trademarks or registered trademarks of other companies.

Note that the trademark symbol "™" and the registered trademark symbol "®" may or may not be used in this document.

IMPORTANT!

1. No part of this document may be copied or reprinted, in whole or in part, without written permission.
2. The contents of this document are subject to change without prior notice and without legal obligation.
3. The contents of this document are correct to the best of our knowledge. Please inform us of any ambiguous or erroneous descriptions, missing information, etc.

REVISION RECORD					
REV.	DATE (YYYY-MM)	REASON /AUTHOR	PAGE CHANGED	SER. No.	DOC. PRODUCT.
INI.	2026-07	Ookouchi	-----		BU

Security Patch Package name

SPZM45-20089_202606MR_hotfix_win10_LTSC2021_x64_enu

Applicable Products for the Security Patch

Vantage Galan 3T	MRT-3020	V12.0 SP000* - V12.0 SP0011*
------------------	----------	------------------------------

Steps to Apply the Security Patch

Please follow the instruction of applying patches defined by each of the product Security Patch Package. For details, please contact the customer service for each system.

Verified Security Patch List

KB	MSRC	explanation(en-us)	URL/File Name	url
KB2565063_x86	MS11-025	Security Update for Microsoft Visual C++ 2010 Service Pack 1 Redistributable Package (KB2565063)	vc_redist_x86.exe	https://catalog.s.download.windowsupdate.com/msdownload/update/software/secur/2011/07/vcredist_x86_28c54491be70c38c97849c3d8cfb9d0d3c515cb.exe
KB2565063_x64	MS11-025	Security Update for Microsoft Visual C++ 2010 Service Pack 1 Redistributable Package (KB2565063)	vc_redist_x64.exe	https://catalog.s.download.windowsupdate.com/msdownload/update/software/secur/2011/07/vcredist_x64_15d032d669078aa6f0f7fd1cbf4115a070bd034d.exe
VS2013_x86	n/a	Visual C++ Redistributable Packages for Visual Studio 2013	vc_redist_x86.exe	https://download.visualstudio.microsoft.com/download/pr/10912115/88b8f4396e3edaa58c1350db34cdd751/vcredist_x86.exe
VS2013_x64	n/a	Visual C++ Redistributable Packages for Visual Studio 2013	vc_redist_x64.exe	https://download.visualstudio.microsoft.com/download/pr/10912049/3831217601a8a928c5119d826baf8f7/vcredist_x64.exe
VC2015-2022x86	n/a	Microsoft Visual C++ 2015-2022 Redistributable (x86)	VC_redist.x86.exe	https://download.visualstudio.microsoft.com/download/pr/73aabf2e-9532-4f68-99f7-3247081a619c/0C09F2611660441084CE0DF425C51C11E147E6447963C3690F97E0B25C55ED64/VC_redist.x86.exe
VC2015-2022x64	n/a	Microsoft Visual C++ 2015-2022 Redistributable (x64)	VC_redist.x64.exe	https://download.visualstudio.microsoft.com/download/pr/73aabf2e-9532-4f68-99f7-3247081a619c/CC0FF0EB1DC3F5188AE6300FAEF32BF5BEEBA4BDD6E8E445A9184072096B713B/VC_redist.x64.exe

KB	MSRC	explanation(en-us)	URL/File Name	url
Edge_x64_149.0.4022.80	n/a	Microsoft Edge-Stable Channel Version 149 Update for x64 based Editions (Build 149.0.4022.80)	microsofedgeenterprisex64.cab	https://catalog.s.download.windowsupdate.com/d/msdownload/update/software/updt/2026/06/microsofedgeenterprisex64_1868acf3158f14ddc080d8e1b60a42a4608481d4.cab
ASP.NETCore_8.0.28	n/a	2026-06 ASP.NET Core 8.0 Runtime (v8.0.28) - Windows x64 Installer	aspnetcore-runtime-8.0.28-win-x64.exe	https://builds.dotnet.microsoft.com/dotnet/aspnetcore/Runtime/8.0.28/aspnetcore-runtime-8.0.28-win-x64.exe
DotNetRuntime_8.0.28	n/a	2026-06 .NET 8.0 Runtime (v8.0.28) - Windows x64 Installer	dotnet-runtime-8.0.28-win-x64.exe	https://builds.dotnet.microsoft.com/dotnet/Runtime/8.0.28/dotnet-runtime-8.0.28-win-x64.exe
KB5031539	n/a	2023-10 Servicing Stack Update for Windows 10 Version 21H2 for x64-based Systems (KB5031539)	ssu-19041.3562-x64_de23c91f483b2e609cec3e4a995639d13205f867.msu	https://catalog.s.download.windowsupdate.com/c/msdownload/update/software/secu/2023/10/ssu-19041.3562-x64_de23c91f483b2e609cec3e4a995639d13205f867.msu
KB890830_v5.142	n/a	Windows Malicious Software Removal Tool x64 - v5.142 (KB890830)	windows-kb890830-x64-v5.142.exe	https://catalog.s.download.windowsupdate.com/d/msdownload/update/software/uprl/2026/06/windows-kb890830-x64-v5.142_652a96c2f87eaafa7bd0340ffb606a7e96abd55a.exe
KB5087053	n/a	2026-05 Cumulative Update for .NET Framework 3.5 and 4.8.1 for Windows 10 Version 21H2 for x64 (KB5087053)	windows10.0-kb5087053-x64-ndp481_542bd2b99c74b7f2dd50e8283e0528b160058f91.msu	https://catalog.s.download.windowsupdate.com/c/msdownload/update/software/secu/2026/05/windows10.0-kb5087053-x64-ndp481_542bd2b99c74b7f2dd50e8283e0528b160058f91.msu
KB5094127	n/a	2026-06 Cumulative Update for Windows 10 Version 21H2 for x64-based Systems (KB5094127)	windows10.0-kb5094127-x64_6cc28e1a08d05bf15d7bf761ac6c4abb69b6d23e.msu	https://catalog.s.download.windowsupdate.com/d/msdownload/update/software/secu/2026/06/windows10.0-kb5094127-x64_6cc28e1a08d05bf15d7bf761ac6c4abb69b6d23e.msu



LEGAL MANUFACTURER

CANON INC.

30-2, SHIMOMARUKO 3-CHOME, OHTA-KU, TOKYO 146-8501, JAPAN