

**TOSHIBA**

**DICOM CONFORMANCE STATEMENT  
FOR  
DIAGNOSTIC ULTRASOUND SYSTEM**

**MODEL SSA-640A *Viamo*<sup>TM</sup> V1.20**

**TOSHIBA MEDICAL SYSTEMS CORPORATION**

© TOSHIBA MEDICAL SYSTEMS CORPORATION 2009  
ALL RIGHTS RESERVED

## **Trademarks**

Viamo is a trademark of Toshiba Medical Systems Corporation.  
This document may include trademarks or registered trademarks of other companies.

### **IMPORTANT!**

- (1) No part of this document may be copied or reprinted, in whole or in part, without written permission.
- (2) The contents of this document are subject to change without prior notice and without our legal obligation.

\*

## 1. CONFORMANCE STATEMENT OVERVIEW

Table 1-1 provides an overview of the network services supported by *Viamo™*.

**Table 1-1  
NETWORK SERVICES**

<b>SOP Classes</b>	<b>User of Service (SCU)</b>	<b>Provider of Service (SCP)</b>
<b>Transfer</b>		
Secondary Capture Image Storage	Yes	No
Ultrasound Image Storage	Yes	No
Ultrasound Multi-frame Image Storage	Yes	No

## 2. TABLE OF CONTENTS

<b>1.</b>	<b>CONFORMANCE STATEMENT OVERVIEW .....</b>	<b>i</b>
<b>2.</b>	<b>TABLE OF CONTENTS .....</b>	<b>a</b>
<b>3.</b>	<b>INTRODUCTION .....</b>	<b>1</b>
3.1	AUDIENCE .....	1
3.2	REMARKS .....	1
3.3	DEFINITIONS, TERMS AND ABBREVIATIONS .....	1
3.4	REFERENCES .....	1
<b>4.</b>	<b>NETWORKING .....</b>	<b>2</b>
4.1	<b>IMPLEMENTATION MODEL .....</b>	<b>2</b>
4.1.1	Application Data Flow .....	2
4.1.2	Functional Definition of AEs .....	2
4.1.3	Sequencing of Real-World Activities .....	3
4.2	<b>AE SPECIFICATIONS .....</b>	<b>4</b>
4.2.1	Verification SCU AE Specification .....	4
4.2.2	Verification SCP AE Specification .....	7
4.2.3	Storage SCU AE Specification .....	10
4.3	<b>NETWORK INTERFACES .....</b>	<b>13</b>
4.3.1	Physical Network Interface .....	13
4.3.2	Additional Protocols .....	13
4.4	<b>CONFIGURATION .....</b>	<b>14</b>
4.4.1	AE Title/Presentation Address Mapping .....	14
4.4.2	Parameters .....	14
<b>5.</b>	<b>MEDIA INTERCHANGE .....</b>	<b>16</b>
<b>6.</b>	<b>SUPPORT OF CHARACTER SETS .....</b>	<b>17</b>
<b>7.</b>	<b>SECURITY .....</b>	<b>18</b>
<b>8.</b>	<b>ANNEXES .....</b>	<b>19</b>
8.1	<b>IOD CONTENTS .....</b>	<b>19</b>
8.1.1	Created SOP Instances .....	19
8.1.2	Usage of Attributes from received IOD's .....	27
8.1.3	Attribute Mapping .....	27
8.1.4	Coerced/Modified Fields .....	27
8.2	<b>DATA DICTIONARY OF PRIVATE ATTRIBUTES .....</b>	<b>27</b>
8.3	<b>CONTROLLED TERMINOLOGY AND TEMPLATES .....</b>	<b>27</b>
8.4	<b>GRAYSCALE IMAGE CONSISTENCY .....</b>	<b>27</b>
8.5	<b>STANDARD EXTENDED/SPECIALIZED/PRIVATE SOP CLASSES .....</b>	<b>27</b>
8.6	<b>PRIVATE TRANSFER SYNTAXES .....</b>	<b>27</b>

## 3. INTRODUCTION

### 3.1 AUDIENCE

This document is intended for hospital staff, health system integrators, software designers or implementers. It is assumed that the reader has a working understanding of DICOM.

### 3.2 REMARKS

DICOM, by itself, does not guarantee interoperability. However, the Conformance Statement facilitates a first-level validation for interoperability between different applications supporting the same DICOM functionality.

This Conformance Statement is not intended to replace validation with other DICOM equipment to ensure proper exchange of information intended.

The scope of this Conformance Statement is to facilitate communication with Toshiba Medical Systems and other vendors' Medical equipment. The Conformance Statement should be read and understood in conjunction with the DICOM Standard [DICOM]. However, by itself it is not guaranteed to ensure the desired interoperability and a successful interconnectivity.

The user should be aware of the following important issues:

- The comparison of different conformance statements is the first step towards assessing interconnectivity between Toshiba Medical Systems and non-Toshiba Medical Systems equipment.
- Test procedures should be defined to validate the desired level of connectivity.
- The DICOM standard will evolve to meet the users' future requirements. Toshiba Medical Systems is actively involved in developing the standard further and therefore reserves the right to make changes to its products or to discontinue its delivery.

### 3.3 DEFINITIONS, TERMS AND ABBREVIATIONS

Definitions, terms and abbreviations used in this document are defined within the different parts of the DICOM standard.

Abbreviations and terms are as follows:

<b>AE</b>	Application Entity
<b>ASCE</b>	Association Control Service Element
<b>DIMSE</b>	DICOM Message Service Element
<b>IE</b>	Information Entity
<b>IOD</b>	Information Object Definition
<b>ISO</b>	International Standard Organization
<b>PDU</b>	Protocol Data Unit
<b>SCU</b>	Service Class User (DICOM client)
<b>SCP</b>	Service Class Provider (DICOM server)
<b>SOP</b>	Service-Object Pair
<b>UID</b>	Unique Identifier


### 3.4 REFERENCES

[DICOM] Digital Imaging and Communications in Medicine (DICOM), NEMA PS 3.1-3.18, 2008

## 4. NETWORKING

### 4.1 IMPLEMENTATION MODEL

#### 4.1.1 Application Data Flow

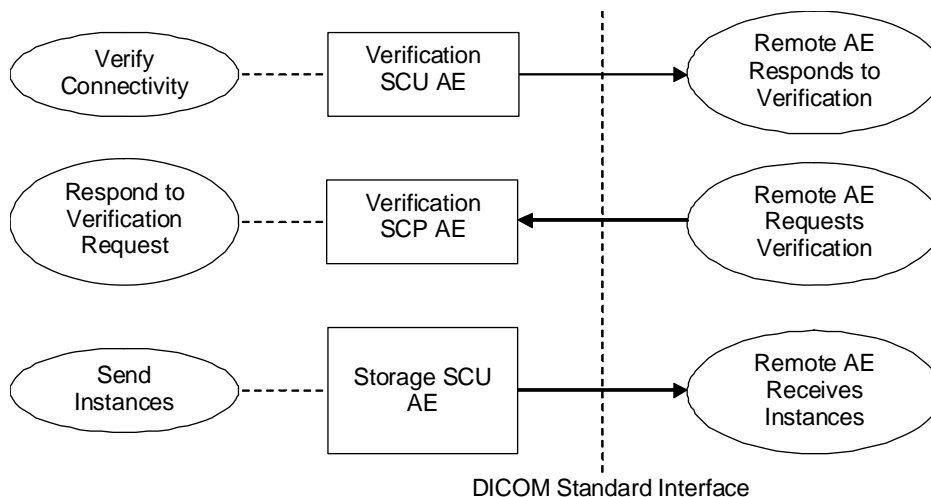

**Figure 4.1-1**  
**APPLICATION DATA FLOW DIAGRAM**

- The Verification SCU AE issues a C-ECHO to verify a DICOM connection to a remote AE. It is associated with the local real-world activity “Verify Connectivity”. “Verify Connectivity” is performed via the Service Tool.
- The Verification SCP AE responds successfully to C-ECHO requests from known AE Titles. It is associated with the local real-world activity “Respond to Verification Request”
- The Storage SCU AE sends instances to a remote AE. It is associated with the local real-world activity “Send Instances”. “Send Instances” is performed upon user request for specific instances selected.

#### 4.1.2 Functional Definition of AEs

##### 4.1.2.1 Functional Definition of Verification SCU AE

The Verification SCU AE issues a C-ECHO to verify a DICOM connection to a remote AE. It is performed via the Service Tool.


##### 4.1.2.2 Functional Definition of Verification SCP AE

The Verification SCP AE responds successfully to C-ECHO requests from known AE Titles.

##### 4.1.2.3 Functional Definition of Storage SCU AE

The existence of a send-job queue entry with associated network destination will activate the Storage SCU AE. An association request is sent to the destination AE and upon successful negotiation of a Presentation Context the image transfer is started. If the image transfer fails, the Storage SCU AE will retry this send-job automatically.

### 4.1.3 Sequencing of Real-World Activities

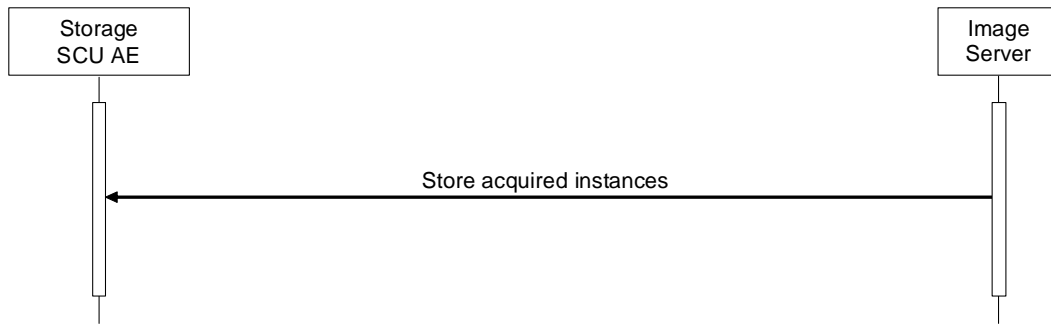

**Figure 4.1-2**  
**SEQUENCING CONSTRAINTS**

## 4.2 AE SPECIFICATIONS

### 4.2.1 Verification SCU AE Specification

#### 4.2.1.1 SOP Classes

The Verification SCU AE provides Standard Conformance to the following SOP Classes:

**Table 4.2-1**  
**SOP CLASSES FOR THE VERIFICATION SCU AE**

SOP Class Name	SOP Class UID	SCU	SCP
Verification	1.2.840.10008.1.1	Yes	No

#### 4.2.1.2 Association Policies

##### 4.2.1.2.1 General

The DICOM standard application context name for DICOM 3.0 is always proposed:

**Table 4.2-2**  
**DICOM APPLICATION CONTEXT FOR THE VERIFICATION SCU AE**

Application Context Name	1.2.840.10008.3.1.1.1
--------------------------	-----------------------

##### 4.2.1.2.2 Number of Associations

The Verification SCU AE initiates one association at a time.

**Table 4.2-3**  
**NUMBER OF ASSOCIATIONS INITIATED FOR THE VERIFICATION SCU AE**

Maximum number of simultaneous associations	1
---	---

##### 4.2.1.2.3 Asynchronous Nature

The Verification SCU AE does not support asynchronous communication (multiple outstanding transactions over a single association).

**Table 4.2-4**  
**ASYNCHRONOUS NATURE FOR THE VERIFICATION SCU AE**

Maximum number of outstanding asynchronous transactions	1
---	---

##### 4.2.1.2.4 Implementation Identifying Information

The implementation information for the Verification SCU AE is:

**Table 4.2-5**  
**DICOM IMPLEMENTATION CLASS AND VERSION FOR THE VERIFICATION SCU AE**

Implementation Class UID	1.2.392.200036.9116.6.17.1000.1
Implementation Version Name	TM_VIAMO_1.0




### 4.2.1.3 Association Initiation Policy

#### 4.2.1.3.1 Activity – Verify Connectivity

##### 4.2.1.3.1.1 Description and Sequencing of Activities

The Verification SCU AE attempts to initiate a new association in order to issue a verification request (C-ECHO) if needed.

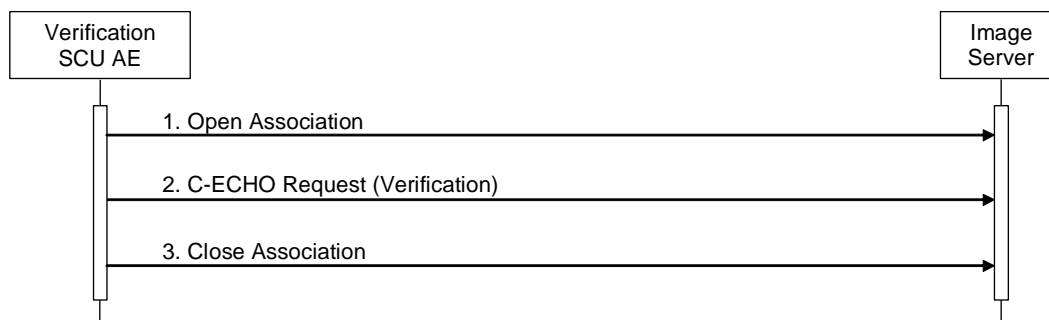

**Figure 4.2-1**  
**SEQUENCING OF ACTIVITY – VERIFY CONNECTIVITY**

A possible sequence of interactions between the Verification SCU AE and an Image Server (e.g. a storage or archive device supporting the Verification SOP Classes as an SCP) is illustrated in the Figure above:

1. The Verification SCU AE opens an association with the Image Server.
2. The Verification SCU AE issues a verification request (C-ECHO) and the Image Server replies with a C-ECHO response (status success).
3. The Verification SCU AE closes the association with the Image Server.

##### 4.2.1.3.1.2 Proposed Presentation Contexts

The Verification SCU AE will propose the Presentation Contexts shown in the following table:

**Table 4.2-6**  
**PROPOSED PRESENTATION CONTEXTS FOR ACTIVITY VERIFY CONNECTIVITY**

Presentation Context Table					
Abstract Syntax		Transfer Syntax		Role	Ext. Neg.
Name	UID	Name List	UID List		
Verification	1.2.840.10008.1.1	Implicit VR Little Endian	1.2.840.10008.1.2	SCU	None
		Explicit VR Little Endian	1.2.840.10008.1.2.1		

#### 4.2.1.3.1.3 SOP Specific Conformance for Verification SOP Class

The Verification SCU AE provides standard conformance to the Verification Service Class as an SCU.

The behavior of Verification SCU AE when encountering status codes in a C-ECHO response is summarized in the table below:

**Table 4.2-7  
VERIFICATION RESPONSE STATUS HANDLING BEHAVIOR**

<b>Service Status</b>	<b>Further Meaning</b>	<b>Status Code</b>	<b>Behavior</b>
Success	Success	0000	The Verification SCU AE judges the remote AE is present and active on the network.

The behavior of Verification SCU AE during communication failure is summarized in the table below:

**Table 4.2-8  
VERIFICATION COMMUNICATION FAILURE BEHAVIOR**

<b>Exception</b>	<b>Behavior</b>
Timeout	The association is aborted and the failure reason is logged and reported to the user.
Association aborted by the SCP or network layers	The failure reason is logged and reported to the user.

## 4.2.2 Verification SCP AE Specification

### 4.2.2.1 SOP Classes

The Verification SCP AE provides Standard Conformance to the following SOP Classes:

**Table 4.2-9**  
**SOP CLASSES FOR THE VERIFICATION SCP AE**

SOP Class Name	SOP Class UID	SCU	SCP
Verification	1.2.840.10008.1.1	No	Yes

### 4.2.2.2 Association Policies

#### 4.2.2.2.1 General

The DICOM standard application context name for DICOM 3.0 is always proposed:

**Table 4.2-10**  
**DICOM APPLICATION CONTEXT FOR THE VERIFICATION SCP AE**

Application Context Name	1.2.840.10008.3.1.1.1
--------------------------	-----------------------

#### 4.2.2.2.2 Number of Associations

**Table 4.2-11**  
**NUMBER OF ASSOCIATIONS ACCEPTED FOR THE VERIFICATION SCP AE**

Maximum number of simultaneous associations	Unlimited
---	-----------

#### 4.2.2.2.3 Asynchronous Nature

The Verification SCP AE does not support asynchronous communication (multiple outstanding transactions over a single association).

**Table 4.2-12**  
**ASYNCHRONOUS NATURE FOR THE VERIFICATION SCP AE**

Maximum number of outstanding asynchronous transactions	1
---	---

#### 4.2.2.2.4 Implementation Identifying Information

The implementation information for the Verification SCP AE is:

**Table 4.2-13**  
**DICOM IMPLEMENTATION CLASS AND VERSION FOR THE VERIFICATION SCP AE**

Implementation Class UID	1.2.392.200036.9116.6.17.1000.1
Implementation Version Name	TM_VIAMO_1.0

### 4.2.2.3 Association Initiation Policy


The Verification SCP AE does not initiate associations.

### 4.2.2.4 Association Acceptance Policy

#### 4.2.2.4.1 Activity – Respond to Verification Request

##### 4.2.2.4.1.1 Description and Sequencing of Activities

When the Verification SCP AE accepts an association, it will respond to a verification request (C-ECHO).

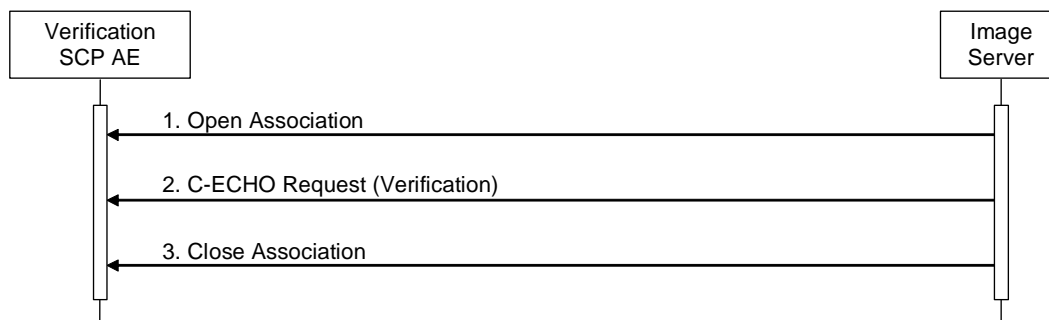

**Figure 4.2-2**  
**SEQUENCING OF ACTIVITY – RESPOND TO VERIFICATION REQUEST**

A possible sequence of interactions between the Verification SCP AE and an Image Server (e.g. a storage or archive device supporting the Verification SOP Classes as an SCU) is illustrated in the Figure above:

1. The Image Server opens an association with the Verification SCP AE.
2. The Image Server issues a verification request (C-ECHO) and the Verification SCP AE replies with a C-ECHO response (status success).
3. The Image Server closes the association with the Verification SCP AE.

The Verification SCP AE may reject association attempts as shown in the table below. The Result, Source and Reason/Diag columns represent the values returned in the appropriate fields of an ASSOCIATE-RJ PDU (see PS 3.8, Section 9.3.4). The contents of the Source column is abbreviated to save space and the meaning of the abbreviations are:

**Table 4.2-14**  
**ASSOCIATION REJECTION REASONS**

Result	Source	Reason/Diag	Explanation
1 – rejected-permanent	DICOM UL service-user	3 – calling-AE-title-not-recognized	The association request contained an unrecognized calling AE Title. An association request with the same parameters will not succeed at a later time unless configuration changes are made. This rejection reason normally occurs when the association acceptor has not been configured to recognize the AE Title of the association initiator.
1 – rejected-permanent	DICOM UL service-provider (ASCE related function)	1 – no-reason-given	The association request could not be parsed. An association request with the same format will not succeed at a later time.

#### 4.2.2.4.1.2 Accepted Presentation Contexts

The default behavior of the Verification SCP AE supports the Implicit VR Little Endian and Explicit VR Little Endian transfer syntaxes. If the both transfer syntaxes are proposed per presentation context then the Verification SCP AE will select Explicit VR Little Endian transfer syntax.

**Table 4.2-15**

#### **PROPOSED PRESENTATION CONTEXTS FOR ACTIVITY RESPOND TO VERIFICATION REQUEST**

<b>Presentation Context Table</b>					
<b>Abstract Syntax</b>		<b>Transfer Syntax</b>		<b>Role</b>	<b>Ext. Neg.</b>
<b>Name</b>	<b>UID</b>	<b>Name List</b>	<b>UID List</b>		
Verification	1.2.840.10008.1.1	Implicit VR Little Endian	1.2.840.10008.1.2	SCP	None
		Explicit VR Little Endian	1.2.840.10008.1.2.1		

#### 4.2.2.4.1.3 SOP Specific Conformance for Verification SOP Class

The Verification SCP AE provides standard conformance to the Verification Service Class as an SCP.

## 4.2.3 Storage SCU AE Specification

### 4.2.3.1 SOP Classes

The Storage SCU AE provides Standard Conformance to the following SOP Classes:

**Table 4.2-16**  
**SOP CLASSES FOR THE STORAGE SCU AE**

SOP Class Name	SOP Class UID	SCU	SCP
Secondary Capture Image Storage	1.2.840.10008.5.1.4.1.1.7	Yes	No
Ultrasound Image Storage	1.2.840.10008.5.1.4.1.1.6.1		
Ultrasound Multi-frame Image Storage	1.2.840.10008.5.1.4.1.1.3.1		

### 4.2.3.2 Association Policies

#### 4.2.3.2.1 General

The DICOM standard application context name for DICOM 3.0 is always proposed:

**Table 4.2-17**  
**DICOM APPLICATION CONTEXT FOR THE STORAGE SCU AE**

Application Context Name	1.2.840.10008.3.1.1.1
--------------------------	-----------------------

#### 4.2.3.2.2 Number of Associations

The Storage SCU AE initiates one association at a time for a transfer request.

**Table 4.2-18**  
**NUMBER OF ASSOCIATIONS INITIATED FOR THE STORAGE SCU AE**

Maximum number of simultaneous associations	1
---	---

#### 4.2.3.2.3 Asynchronous Nature

The Storage SCU AE does not support asynchronous communication (multiple outstanding transactions over a single association).

**Table 4.2-19**  
**ASYNCHRONOUS NATURE FOR THE STORAGE SCU AE**

Maximum number of outstanding asynchronous transactions	1
---	---

#### 4.2.3.2.4 Implementation Identifying Information

The implementation information for the Storage SCU AE is:

**Table 4.2-20**  
**DICOM IMPLEMENTATION CLASS AND VERSION FOR THE STORAGE SCU AE**


Implementation Class UID	1.2.392.200036.9116.6.17.1000.1
Implementation Version Name	TM_VIAMO_1.0

### 4.2.3.3 Association Initiation Policy

#### 4.2.3.3.1 Activity – Send Images

##### 4.2.3.3.1.1 Description and Sequencing of Activities

The Storage SCU AE attempts to initiate a new association in order to issue a storage request (C-STORE). If the image transfer fails, the Storage SCU AE will retry this send-job automatically.

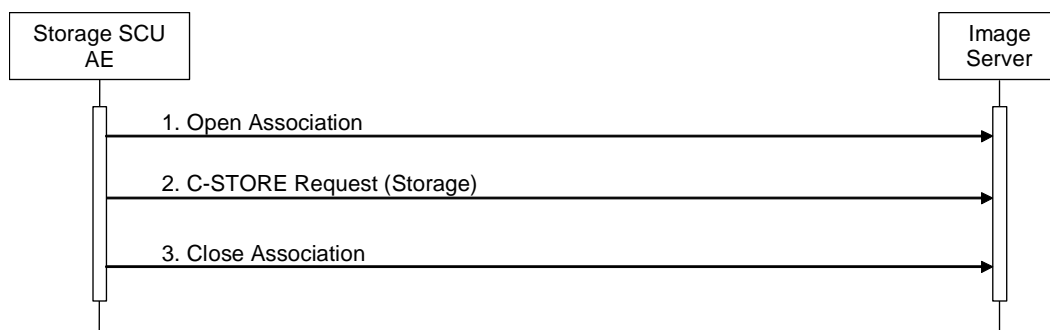

**Figure 4.2-3**  
**SEQUENCING OF ACTIVITY – SEND IMAGES**

A possible sequence of interactions between the Storage SCU AE and an Image Server (e.g. a storage or archive device supporting the Storage SOP Classes as an SCP) is illustrated in the Figure above:

1. The Storage SCU AE opens an association with the Image Server.
2. Acquired images are transmitted to the Image Server using a storage request (C-STORE) and the Image Server replies with a C-STORE response (status success).
3. The Storage SCU AE closes the association with the Image Server.

##### 4.2.3.3.1.2 Proposed Presentation Contexts

The Storage SCU AE will propose the Presentation Contexts shown in the following table:

**Table 4.2-21**  
**PROPOSED PRESENTATION CONTEXTS FOR ACTIVITY SEND IMAGES**

Presentation Context Table					
Abstract Syntax		Transfer Syntax		Role	Ext. Neg.
Name	UID	Name List	UID List		
Secondary Capture Image Storage	1.2.840.10008.5.1.4.1.1.7	Implicit VR Little Endian	1.2.840.10008.1.2	SCU	None
		Explicit VR Little Endian	1.2.840.10008.1.2.1		
		JPEG Baseline (Process 1)	1.2.840.10008.1.2.4.50		
Ultrasound Image Storage	1.2.840.10008.5.1.4.1.1.6.1	Implicit VR Little Endian	1.2.840.10008.1.2		
		Explicit VR Little Endian	1.2.840.10008.1.2.1		
		JPEG Baseline (Process 1)	1.2.840.10008.1.2.4.50		
Ultrasound Multi-frame Image Storage	1.2.840.10008.5.1.4.1.1.3.1	Implicit VR Little Endian	1.2.840.10008.1.2		
		Explicit VR Little Endian	1.2.840.10008.1.2.1		
		JPEG Baseline (Process 1)	1.2.840.10008.1.2.4.50		

#### 4.2.3.3.1.3 SOP Specific Conformance for Storage SOP Classes

The Storage SCU AE provides standard conformance to the Storage Service Class as an SCU.

The behavior of Storage SCU AE when encountering status codes in a C-STORE response is summarized in the table below:

**Table 4.2-22**  
**STORAGE C-STORE RESPONSE STATUS HANDLING BEHAVIOR**

Service Status	Further Meaning	Status Code	Behavior
Success	Success	0000	The SCP has successfully stored the SOP Instance. If all SOP Instances in a send job have status success then the job is marked as complete.
Refused	Out of Resources	A7xx	The association is aborted and the send job is marked as failed. The status meaning is logged and the job failure is reported to the user via the job control application.
Error	Data Set does not match SOP Class	A9xx	
Error	Cannot Understand	Cxxx	
Warning	Coercion of Data Elements	B000	
Warning	Data Set does not match SOP Class	B007	
Warning	Elements Discarded	B006	
*	*	Any other status code	

The behavior of Storage SCU AE during communication failure is summarized in the table below:

**Table 4.2-23**  
**STORAGE COMMUNICATION FAILURE BEHAVIOR**

Exception	Behavior
Timeout	The association is aborted and the send job is marked as failed. The reason is logged and the job failure is reported to the user via the job control application.
Association aborted by the SCP or network layers	The send job is marked as failed. The reason is logged and the job failure is reported to the user via the job control application.

If the image transfer fails, the Storage SCU AE will retry this send-job automatically. The number of retries is configurable.

The contents of Image Storage SOP Instances created by the Storage SCU AE conform to the DICOM Image IOD definitions and are described in section 8.1.



## 4.3 NETWORK INTERFACES

### 4.3.1 Physical Network Interface

This product supports a single network interface. One of the following physical network interfaces will be available depending on installed hardware options:

**Table 4.3-1**  
**SUPPORTED PHYSICAL NETWORK INTERFACES**

Ethernet 1000baseT
Ethernet 100baseT
Ethernet 10baseT

### 4.3.2 Additional Protocols

None.

## 4.4 CONFIGURATION

### 4.4.1 AE Title/Presentation Address Mapping

#### 4.4.1.1 Local AE Titles

All local applications use the AE Titles and TCP/IP Ports configured via the Service Tool. The Field Service Engineer can configure the TCP Port via the Service Tool.

Note: Up to 16 characters (alphanumeric characters, "-", ".", and "\_") can be used in the AE Titles.

**Table 4.4-1  
AE TITLE CONFIGURATION TABLE**

Application Entity	Default AE Title	Default TCP/IP Port
Verification SCP	DICOM_LOCAL_SCP	104
Verification SCU	VERIFYSCU_AE	Not Applicable
Storage SCU	DICOM_LOCAL_SCU	

#### 4.4.1.2 Remote AE Title/Presentation Address Mapping

The AE Titles, host names and port numbers of remote applications are configured using the Service Tool.

Note: Up to 16 characters (alphanumeric characters, "-", ".", and "\_") can be used in the AE Titles.

### 4.4.2 Parameters

A large number of parameters related to acquisition and general operation can be configured using the Service Tool. The table below only shows those configuration parameters relevant to DICOM communication. See the Product's Service Manual for details on general configuration capabilities.

**Table 4.4-2  
CONFIGURATION PARAMETERS TABLE**

Parameter	Configurable (Yes/No)[Range]	Default Value
<b>General Parameters</b>		
Time-out waiting for an acceptance or rejection response to an association request (Application Level timeout)	Yes	240 sec
Time-out waiting for a response to an association release request (Application Level timeout)	Yes	240 sec
Time-out waiting for completion of a TCP/IP connect request (Low-level timeout)	Yes	240 sec
Time-out awaiting a response to a DIMSE request (Low-level timeout)	Yes	240 sec
Time-out for waiting for data between TCP/IP-packets (Low-level timeout)	Yes	240 sec

Parameter	Configurable (Yes/No)[Range]	Default Value
<b>Storage SCU Parameters</b>		
Maximum number of simultaneously initiated associations by the Storage SCU AE	No	1
Supported transfer syntaxes (separately configurable for each remote AE)	Yes	Implicit VR Little Endian Explicit VR Little Endian JPEG Baseline (Process 1)
Number of times a failed send job may be retried	Yes	3

## **5. MEDIA INTERCHANGE**

This product does not support Media Storage.

## 6. SUPPORT OF CHARACTER SETS

This product supports the following character sets:

- ISO-IR 6 (default)                      ISO 646
- ISO-IR 100 (Latin alphabet No.1)    Supplementary set of ISO 8859

## 7. SECURITY

This product does not support any specific security measures.

It is assumed that the product is used within a secured environment. It is assumed that a secured environment includes at a minimum:

- a. Firewall or router protections to ensure that only approved external hosts have network access to the product.
- b. Firewall or router protections to ensure that the product only has network access to approved external hosts and services.

Other network security procedures such as automated intrusion detection may be appropriate in some environments. Additional security features may be established by the local security policy and are beyond the scope of this conformance statement.

## 8. ANNEXES

### 8.1 IOD CONTENTS

#### 8.1.1 Created SOP Instances

Table 8.1-1 specifies the attributes of a Secondary Capture Image transmitted by the Storage SCU AE.

Table 8.1-2 specifies the attributes of an Ultrasound Image transmitted by the Storage SCU AE.

Table 8.1-3 specifies the attributes of an Ultrasound Multi-frame Image transmitted by the Storage SCU AE.

The following tables use a number of abbreviations. The abbreviations used in the “Presence of ...” column are:

VNAP	Value Not Always Present (attribute sent zero length if no value is present)
ANAP	Attribute Not Always Present
ALWAYS	Always Present
EMPTY	Attribute is sent without a value

The abbreviations used in the “Source” column:

MWL	the attribute value source Modality Worklist
USER	the attribute value source is from user input
AUTO	the attribute value is generated automatically
MPPS	the attribute value is the same as that use for Modality Performed Procedure Step
CONFIG	the attribute value source is a configurable parameter

## 8.1.1.1 SC Image IOD

**Table 8.1-1  
IOD OF CREATED SC IMAGE SOP INSTANCES**

<b>IE</b>	<b>Module</b>	<b>Reference</b>	<b>Presence of Module</b>
Patient	Patient	Table 8.1-4	ALWAYS
	Clinical Trial Subject	--	Not Present
Study	General Study	Table 8.1-5	ALWAYS
	Patient Study	Table 8.1-6	ALWAYS
	Clinical Trial Study	--	Not Present
Series	General Series	Table 8.1-7	ALWAYS
	Clinical Trial Series	--	Not Present
Equipment	General Equipment	Table 8.1-8	ALWAYS
	SC Equipment	Table 8.1-11	ALWAYS
Image	General Image	Table 8.1-9	ALWAYS
	Image Pixel	Table 8.1-10	ALWAYS
	SC Image	--	Not Present
	Overlay Plane	--	Not Present
	Modality LUT	--	Not Present
	VOI LUT	--	Not Present
	SOP Common	Table 8.1-12	ALWAYS



## 8.1.1.2 US Image IOD

**Table 8.1-2  
IOD OF CREATED US IMAGE SOP INSTANCES**

<b>IE</b>	<b>Module</b>	<b>Reference</b>	<b>Presence of Module</b>
Patient	Patient	Table 8.1-4	ALWAYS
	Clinical Trial Subject	--	Not Present
Study	General Study	Table 8.1-5	ALWAYS
	Patient Study	Table 8.1-6	ALWAYS
	Clinical Trial Study	--	Not Present
Series	General Series	Table 8.1-7	ALWAYS
	Clinical Trial Series	--	Not Present
Frame of Reference	Frame of Reference	--	Not Present
	Synchronization	--	Not Present
Equipment	General Equipment	Table 8.1-8	ALWAYS
Image	General Image	Table 8.1-9	ALWAYS
	Image Pixel	Table 8.1-10	ALWAYS
	Contrast/bolus	--	Not Present
	Palette Color Lookup Table	--	Not Present
	US Region Calibration	--	Not Present
	US Image	Table 8.1-13	ALWAYS
	Overlay Plane	--	Not Present
	VOI LUT	--	Not Present
	SOP Common	Table 8.1-14	ALWAYS

## 8.1.1.3 US Multi-frame Image IOD

**Table 8.1-3  
IOD OF CREATED US MULTI-FRAME IMAGE SOP INSTANCES**

<b>IE</b>	<b>Module</b>	<b>Reference</b>	<b>Presence of Module</b>
Patient	Patient	Table 8.1-4	ALWAYS
	Clinical Trial Subject	--	Not Present
Study	General Study	Table 8.1-5	ALWAYS
	Patient Study	Table 8.1-6	ALWAYS
	Clinical Trial Study	--	Not Present
Series	General Series	Table 8.1-7	ALWAYS
	Clinical Trial Series	--	Not Present
Frame of Reference	Frame of Reference	--	Not Present
	Synchronization	--	Not Present
Equipment	General Equipment	Table 8.1-8	ALWAYS
Image	General Image	Table 8.1-9	ALWAYS
	Image Pixel	Table 8.1-10	ALWAYS
	Contrast/bolus	--	Not Present
	Cine	Table 8.1-15	ALWAYS
	Multi-frame	Table 8.1-16	ALWAYS
	Frame Pointers	--	Not Present
	Palette Color Lookup Table	--	Not Present
	US Region Calibration	--	Not Present
	US Image	Table 8.1-17	ALWAYS
	VOI LUT	--	Not Present
	SOP Common	Table 8.1-18	ALWAYS

## 8.1.1.4 Common Modules

**Table 8.1-4  
PATIENT MODULE OF CREATED SOP INSTANCES**

Attribute Name	Tag	VR	Value	Presence of Value	Source
Patient's Name	(0010,0010)	PN		VNAP	USER
Patient ID	(0010,0020)	LO		VNAP	USER
Patient's Birth Date	(0010,0030)	DA		VNAP	USER
Patient's Sex	(0010,0040)	CS		VNAP	USER
Patient Comments	(0010,4000)	LT		ANAP	USER

**Table 8.1-5  
GENERAL STUDY MODULE OF CREATED SOP INSTANCES**

Attribute Name	Tag	VR	Value	Presence of Value	Source
Study Instance UID	(0020,000D)	UI		ALWAYS	AUTO
Study Date	(0008,0020)	DA		ALWAYS	AUTO
Study Time	(0008,0030)	TM		ALWAYS	AUTO
Referring Physician's Name	(0008,0090)	PN		VNAP	USER
Study ID	(0020,0010)	SH		ALWAYS	AUTO
Accession Number	(0008,0050)	SH		VNAP	USER
Study Description	(0008,1030)	LO		ANAP	USER
Study Comments	(0032,4000)	LT	Additional Info from user input will be edited in the following format: <"BSA="BSA Information<LINEFEED> "BloodPressure="Blood Pressure Information<LINEFEED> Additional Info<LINEFEED> "BSAType="BSA Type Information>.	ALWAYS	USER

**Table 8.1-6  
PATIENT STUDY MODULE OF CREATED SOP INSTANCES**

Attribute Name	Tag	VR	Value	Presence of Value	Source
Patient's Size	(0010,1020)	DS		VNAP	USER
Patient's Weight	(0010,1030)	DS		VNAP	USER

**Table 8.1-7  
GENERAL SERIES MODULE OF CREATED SOP INSTANCES**

Attribute Name	Tag	VR	Value	Presence of Value	Source
Modality	(0008,0060)	CS	US	ALWAYS	AUTO
Series Instance UID	(0020,000E)	UI		ALWAYS	AUTO
Series Number	(0020,0011)	IS		ALWAYS	AUTO
Series Date	(0008,0021)	DA		ALWAYS	AUTO
Series Time	(0008,0031)	TM		ALWAYS	AUTO
Performing Physician's Name	(0008,1050)	PN		ANAP	USER
Operator's Name	(0008,1070)	PN		ANAP	USER

**Table 8.1-8**

**GENERAL EQUIPMENT MODULE OF CREATED SOP INSTANCES**

Attribute Name	Tag	VR	Value	Presence of Value	Source
Manufacturer	(0008,0070)	LO	TOSHIBA_MEC	ALWAYS	AUTO
Institution Name	(0008,0080)	LO		ANAP	CONFIG
Institutional Department Name	(0008,1040)	LO		ANAP	USER
Manufacturer's Model Name	(0008,1090)	LO	Viamo	ALWAYS	AUTO
Device Serial Number	(0018,1000)	LO		ALWAYS	AUTO
Software Version	(0018,1020)	LO	V1.0	ALWAYS	AUTO

Table 8.1-9

**GENERAL IMAGE MODULE OF CREATED SOP INSTANCES**

Attribute Name	Tag	VR	Value	Presence of Value	Source
Instance Number	(0020,0013)	IS		ALWAYS	AUTO
Patient Orientation	(0020,0020)	CS		EMPTY	AUTO
Content Date	(0008,0023)	DA		ALWAYS	AUTO
Content Time	(0008,0033)	TM		ALWAYS	AUTO
Image Type	(0008,0008)	CS		ALWAYS	AUTO
Acquisition Date	(0008,0022)	DA		ALWAYS	AUTO
Acquisition Time	(0008,0032)	TM		ALWAYS	AUTO
Image Comments	(0020,4000)	LT		ANAP	AUTO
Lossy Image Compression	(0028,2110)	CS		ANAP	AUTO
Lossy Image Compression Ratio	(0028,2112)	DS		ANAP	AUTO

Table 8.1-10

**IMAGE PIXEL MODULE OF CREATED SOP INSTANCES**

Attribute Name	Tag	VR	Value	Presence of Value	Source
Samples per Pixel	(0028,0002)	US	3	ALWAYS	AUTO
Photometric Interpretation	(0028,0004)	CS	"RGB" or "YBR_FULL_422"	ALWAYS	CONFIG
Planar Configuration	(0028,0006)	US	0	ANAP	AUTO
Rows	(0028,0010)	US	600 or 768	ALWAYS	AUTO
Columns	(0028,0011)	US	800 or 1024	ALWAYS	AUTO
Bits Allocated	(0028,0100)	US	8	ALWAYS	AUTO
Bits Stored	(0028,0101)	US	8	ALWAYS	AUTO
High Bit	(0028,0102)	US	7	ALWAYS	AUTO
Pixel Representation	(0028,0103)	US	0	ALWAYS	AUTO
Pixel Data	(7FE0,0010)	OB or OW		ALWAYS	AUTO

### 8.1.1.5 SC Image Modules

**Table 8.1-11  
SC EQUIPMENT MODULE OF CREATED SC IMAGE SOP INSTANCES**

Attribute Name	Tag	VR	Value	Presence of Value	Source
Conversion Type	(0008,0064)	CS	“DV” (Digitized Video), “DI” (Digital Interface), “DF” (Digitized Film), or “WSD” (Workstation)	ALWAYS	AUTO

**Table 8.1-12  
SOP COMMON MODULE OF CREATED SC IMAGE SOP INSTANCES**

Attribute Name	Tag	VR	Value	Presence of Value	Source
Specific Character Set	(0008,0008)	CS	ISO_IR 100	ALWAYS	AUTO
SOP Class UID	(0008,0016)	UI	1.2.840.10008.5.1.4.1.1.7	ALWAYS	AUTO
SOP Instance UID	(0008,0018)	UI		ALWAYS	AUTO

### 8.1.1.6 US Image Modules

**Table 8.1-13  
US IMAGE MODULE OF CREATED US IMAGE SOP INSTANCES**

Attribute Name	Tag	VR	Value	Presence of Value	Source
Samples per Pixel	(0028,0002)	US	3	ALWAYS	AUTO
Photometric Interpretation	(0028,0004)	CS	“RGB” or “YBR_FULL_422”	ALWAYS	CONFIG
Planar Configuration	(0028,0006)	US	0	ANAP	AUTO
Rows	(0028,0010)	US	600	ALWAYS	AUTO
Columns	(0028,0011)	US	800	ALWAYS	AUTO
Ultrasound Color Data Present	(0028,0014)	US	1	ALWAYS	AUTO
Bits Allocated	(0028,0100)	US	8	ALWAYS	AUTO
Bits Stored	(0028,0101)	US	8	ALWAYS	AUTO
High Bit	(0028,0102)	US	7	ALWAYS	AUTO
Pixel Representation	(0028,0103)	US	0	ALWAYS	AUTO
Pixel Data	(7FE0,0010)	OB or OW		ALWAYS	AUTO

**Table 8.1-14  
SOP COMMON MODULE OF CREATED US IMAGE SOP INSTANCES**

Attribute Name	Tag	VR	Value	Presence of Value	Source
Specific Character Set	(0008,0008)	CS	ISO_IR 100	ALWAYS	AUTO
SOP Class UID	(0008,0016)	UI	1.2.840.10008.5.1.4.1.1.6.1	ALWAYS	AUTO
SOP Instance UID	(0008,0018)	UI		ALWAYS	AUTO

### 8.1.1.7 US Multi-frame Image Modules

**Table 8.1-15  
CINE MODULE OF CREATED US MULTI-FRAME IMAGE SOP INSTANCES**

Attribute Name	Tag	VR	Value	Presence of Value	Source
Start Trim	(0008,2142)	IS		ANAP	AUTO
Stop Trim	(0008,2143)	IS		ANAP	AUTO
Recommended Display Frame Rate	(0008,2144)	IS		ANAP	USER
Cine Rate	(0018,0040)	IS		ANAP	USER
Effective Duration	(0018,0072)	DS		ANAP	AUTO
Frame Time	(0018,1063)	DS		ALWAYS	AUTO
Frame Delay	(0018,1066)	DS		ANAP	AUTO
Actual Frame Duration	(0018,1242)	IS		ANAP	AUTO

**Table 8.1-16  
MULTI-FRAME MODULE OF CREATED US MULTI-FRAME IMAGE SOP INSTANCES**

Attribute Name	Tag	VR	Value	Presence of Value	Source
Number of Frames	(0028,0008)	IS		ALWAYS	USER
Frame Increment Pointer	(0028,0009)	AT		ALWAYS	AUTO

**Table 8.1-17  
US IMAGE MODULE OF CREATED US MULTI-FRAME IMAGE SOP INSTANCES**

Attribute Name	Tag	VR	Value	Presence of Value	Source
Stage Name	(0008,2120)	SH		ANAP	AUTO
Stage Number	(0008,2122)	IS		ANAP	AUTO
Number of Stages	(0008,2124)	IS		ANAP	AUTO
View Name	(0008,2127)	SH		ANAP	AUTO
View Number	(0008,2128)	IS		ANAP	AUTO
Number of Views in Stage	(0008,212A)	IS		ANAP	AUTO
Heart Rate	(0018,1088)	IS		ANAP	AUTO
Samples per Pixel	(0028,0002)	US	3	ALWAYS	AUTO
Photometric Interpretation	(0028,0004)	CS	"RGB" or "YBR_FULL_422"	ALWAYS	AUTO
Planar Configuration	(0028,0006)	US	0	ALWAYS	AUTO
Rows	(0028,0010)	US	600	ALWAYS	AUTO
Columns	(0028,0011)	US	800	ALWAYS	AUTO
Ultrasound Color Data Present	(0028,0014)	US	1	ANAP	AUTO
Bits Allocated	(0028,0100)	US	8	ALWAYS	AUTO
Bits Stored	(0028,0101)	US	8	ALWAYS	AUTO
High Bit	(0028,0102)	US	7	ALWAYS	AUTO
Pixel Representation	(0028,0103)	US	0	ALWAYS	AUTO
Pixel Data	(7FE0,0010)	OB or OW		ALWAYS	AUTO

**Table 8.1-18  
SOP COMMON MODULE OF CREATED US MULTI-FRAME IMAGE SOP INSTANCES**

<b>Attribute Name</b>	<b>Tag</b>	<b>VR</b>	<b>Value</b>	<b>Presence of Value</b>	<b>Source</b>
Specific Character Set	(0008,0008)	CS	ISO_IR 100	ALWAYS	AUTO
SOP Class UID	(0008,0016)	UI	1.2.840.10008.5.1.4.1.1.3.1	ALWAYS	AUTO
SOP Instance UID	(0008,0018)	UI		ALWAYS	AUTO

### **8.1.2 Usage of Attributes from received IOD's**

No SOP Class specific fields are required.

### **8.1.3 Attribute Mapping**

Not applicable.

### **8.1.4 Coerced/Modified Fields**

Not applicable.

## **8.2 DATA DICTIONARY OF PRIVATE ATTRIBUTES**

Not applicable.

## **8.3 CONTROLLED TERMINOLOGY AND TEMPLATES**

Not applicable.

## **8.4 GRAYSCALE IMAGE CONSISTENCY**

Not applicable.

## **8.5 STANDARD EXTENDED/SPECIALIZED/PRIVATE SOP CLASSES**

Not applicable.

## **8.6 PRIVATE TRANSFER SYNTAXES**

Not applicable.